# Kinetec Maestra™

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Contents

	Page
<b>1. Definition .....</b>	<b>2</b>
<b>2. Indications .....</b>	<b>2</b>
<b>3. Clinical Benefits .....</b>	<b>2</b>
<b>4. Contraindications .....</b>	<b>2</b>
<b>6. Compliance .....</b>	<b>3</b>
<b>7. Unpacking and packing .....</b>	<b>3</b>
<b>8. Installing the device.....</b>	<b>3</b>
<b>9. Description .....</b>	<b>4</b>
<b>10. Electrical connection.....</b>	<b>5</b>
<b>11. Procedure to start the machine .....</b>	<b>5</b>
<b>12. Using the hand control.....</b>	<b>5</b>
12.1. Changing the display language .....	5
12.2. START / STOP / REVERSE function .....	6
12.3. Procedure to stop the machine .....	6
12.4. Locking - Unlocking the hand control setting .....	6
12.5. Possible values for each parameter .....	6
12.6. Quick Start .....	7
12.7. To adjust the basic parameters of the movement .....	8
12.8. PROGRAM MODE.....	9
12.8.1. To enter a program.....	9
12.8.2. Using Programs .....	10
12.8.3. Reading the values of a program: e.g. SPEED .....	10
12.8.4. Program table: .....	11
12.9. To define the upper and lower movement limits.....	12
12.9.1. At the start of a session .....	12
12.9.2. During the session.....	12
<b>13. Mechanical adjustments .....</b>	<b>13</b>
13.1. Setting up the forearm support.....	13
13.2. Setting up the driving mechanism .....	13
13.3. Adjusting the finger length .....	14
<b>14. Possible movements .....</b>	<b>15</b>
14.1. Pronation / Supination.....	15
14.2. Extension / Flexion of the wrist .....	15
14.3. Extension / Flexion of the wrist and MP joints of the fingers.....	16
14.4. Ulnar deviation .....	16
14.5. Thumb opposition .....	17
14.6. Extension / Flexion of the 3 joints of the long fingers .....	18
14.7. Intrinsic + .....	18
14.8. Intrinsic - .....	19
14.9. Extension / Flexion of the DIP joint .....	19
<b>15. Product information.....</b>	<b>20</b>
15.1. Maintenance .....	20
15.2. Troubleshooting guide.....	20
15.3. Cleaning .....	20
15.4. Disposal and recycling.....	20
15.5. Technical specifications.....	21
15.6. Symbols used .....	21
15.7. Warranty .....	21
15.8. Guidance and manufacturer's declaration .....	22

## 1. Definition

The device Kinetec Maestra™ presents the exclusive advantage of offering a large range of options for the passive mobilisation of the various joints of the hand and wrist:

The forearm, wrist, thumb, long fingers.

## 2. Indications

- Tenolysis.
- Aponeurectomies for Dupuytren's disease.
- Interphalangeal and metacarpophalangeal arthroplasties.
- Metacarpophalangeal arthrolyses.
- Tenoarthrolyses.
- Osteosynthesised fractures.
- Rheumatoid or neurological stiffness, or stiffness after burns.
- Tendon sutures.
- Algodystrophies.
- Capsulotomies.
- Synovectomies.

## 3. Clinical Benefits

- Effectively breaks the vicious circle of: trauma → immobility → effusion → atrophy.
- Prevents joint stiffness.
- Speeds the recovery of post-operative range of motion.
- Maintains the quality of the joint surface.
- Promotes joint cartilage healing.
- Provides immediate post-operative continuous passive motion.
- Reduces hospitalization time.
- Reduces the need for pain medication.
- Visual feedback for users.
- Realization of postures (resting, stretching muscles).

## 4. Contraindications

Rheumatoid arthritis in the inflammatory phase, algodystrophy in the inflammatory phase (hyper painful), para-osteo-arthroplasty, unhealed infected wounds, bone cancer, infectious arthritis, deformed joint surfaces, paralysed limbs (atonic or spastic), non-stabilised fractures.

## 5. Warning and safety instructions

- WARNING: The machine must be installed and commissioned according to the information provided in this manual.
- WARNING: If you need any assistance in the assembly, use or maintenance of the device, please contact your KINETEC® distributor.
- WARNING: The practitioner determines the protocol and ensures its proper implementation (settings, session duration and frequency of use).
- WARNING: Run a cycle with the device unloaded before installing the patient on the machine.
- WARNING: For optimum safety, always give the hand control to the patient before starting the system. The patient must know the start/stop/reverse function on the hand control, see page 6.
- WARNING: To avoid the parameters being changed, lock the machine's hand control before giving it to the patient, see page 6.
- WARNING: Danger, risk of explosion: Do not use the machine with anaesthetic gas or in an environment that is rich in oxygen.
- WARNING: For Type B Class I devices, and to avoid all risks of electric shock, the machine should only be connected to a power supply that has protective earthing, see page 5.
- WARNING: Before using this machine, always check that the electrical socket is in good condition and is suitable for the splint power supply cord. Only use the original cable supplied with the machine. Check that the cables remain free around the device so that they do not get damaged.
- WARNING: Before using this machine, always check that the machine is not damaged, in particular the protective housings.
- WARNING: In case of electromagnetic interference with other devices move the device.
- WARNING: Please do not touch the fixed or moving parts while the unit is running: pinching or crushing risk. Keep children and pets away from the machine.
- WARNING: Modifying the machine in any way is strictly forbidden.
- WARNING: Always check the motion parameters displayed on the hand control before starting the device.
- WARNING: Only the accessories, spare parts and supplies described in this manual should be used with this machine.
- WARNING: Do not connect the device to other devices not described in this manual.
- WARNING: If unforeseen events or malfunctions occur, please contact your KINETEC® distributor.
- WARNING: Wireless communications devices, such as domestic wireless devices in networks, mobile phones, wireless telephones and their base stations and walkie-talkies, may affect the machine. You are recommended to keep at least a distance  $d$  between these devices and the machine. See the table on page 23.

## 6. Compliance

The device Kinetec Maestra™ complies with the standards of Directive 93/42/EEC, and bears the EC mark. The device Kinetec Maestra™ complies with the standards in force (IEC 60601-1-2) concerning the electromagnetic compatibility of medical devices and IEC 60601-1 concerning electrical safety. The device Kinetec Maestra™ meets the requirements of the Machinery Directive No. 2006/42/EC.

## 7. Unpacking and packing

### Unpacking

When you unpack the machine, don't forget that you may need to pack it up again. We recommend that you keep the packaging materials, boxes and plastic bags.

**Recommendations for plastic bags: do not put them over the head as there is a risk of suffocation, and keep them out of the reach of children.**

**Be careful with small-sized pieces: they could be swallowed by a child.**

**Be careful with cables and wires: risk of strangulation.**

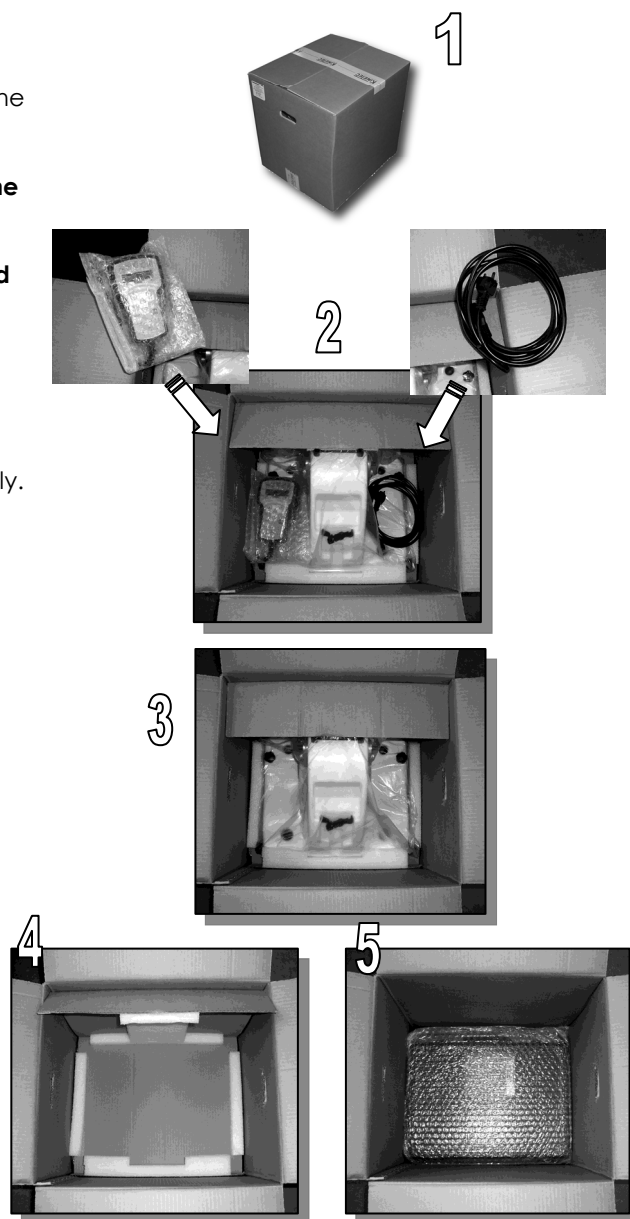
Remove all the components from their packaging.

The accessories case is under the box for the machine. See step 5.

Your machine is ready to be connected to the power supply. (See page 5)

### Packing

To prevent any problems when the machine is transported, only pack it using its original packaging.

## 8. Installing the device

The device Kinetec Maestra™ machine is designed to be used in hospitals, clinics, doctors' offices or in private homes (rental).

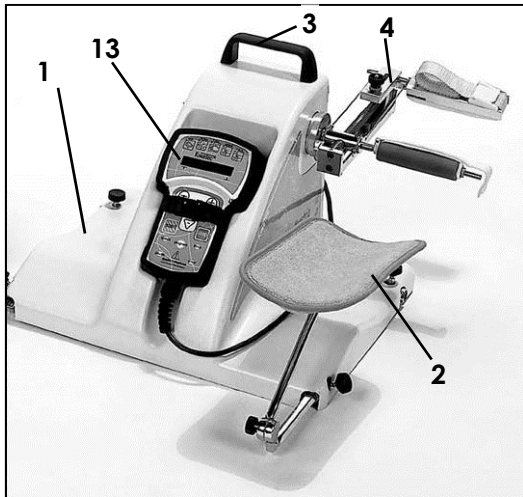
The machine must be installed on a flat surface that is wide enough to accommodate the entire device and possibly the accessory case.

We recommend using the machine with a physiotherapy table in order to benefit from its adjustable height and the large support area.

We do not recommend the use of a chair with castors.

## 9. Description


The device Kinetec Maestra™ is a portable device. It consists of the following components:

- 1 • Casing enclosing the motor and the power PCB.
- 2 • Forearm support.
- 3 • Transport handle.
- 4 • Driving mechanism.

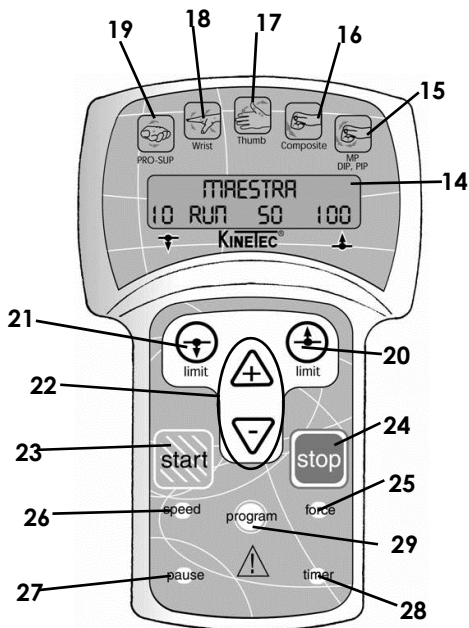
- 5 • Case.
- 6 • Wrist mobilisation accessories.
- 7 • Thumb mobilisation accessories.
- 8 • Long fingers mobilisation accessories.
- 9 • Spare parts.
- 10 • Textile accessories.
- 11 • Ulnar deviation mobilisation accessories.
- 12 • Power supply cable transport position.
- 13 • Hand control.

- 14 • Liquid-crystal display (2 lines of 16 characters).
- 15 • Long fingers selective movements key.
- 16 • Long fingers complete movement key.
- 17 • Thumb opposition movement key.
- 18 • Wrist movement key.
- 19 • Pronation / supination movement key.
- 20 • Upper limits setting.
- 21 • Lower limits setting.
- 22 • Increase / decrease keys.
- 23 • START key.
- 24 • STOP key.
- 25 • FORCE key.
- 26 • SPEED key.
- 27 • PAUSE key.
- 28 • TIMER key.
- 29 • PROGRAM access key.

### Display Details:

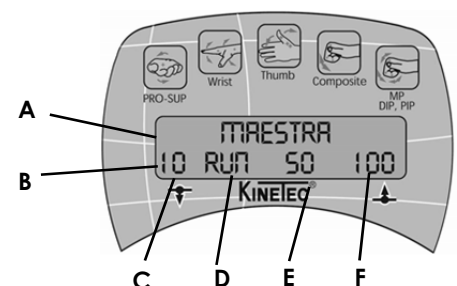- A • 16-character line, used to display various messages when starting up the machine, then the type of movement during operation.
- B • 16-character line, used to display various messages when starting up the machine; then it displays the operational parameters.
- C • 3-character area showing the lower limit of the movement.
- D • 4-character area showing various messages: RUN, STOP, EXT, FLEX ...
- E • 3-character area showing the real-time angle; this value changes with the current movement.
- F • 3-character area showing the upper limit of the movement.

## 10. Electrical connection

2 versions of electrical protection are available for the device Kinetec Maestra™ machine. The identification label shows which version a machine is equipped with:

**Type BF, Class II devices,**

for home use, bear the following symbols:

**Type B class I devices,**

for use in a professional environment, bear the following symbol:

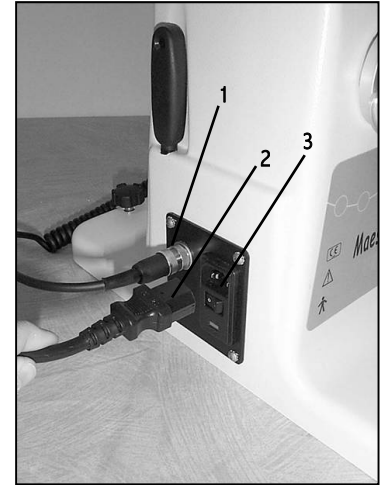
See page 21 for the meaning of the symbols.

Before connecting the device to the power supply, check that the mains voltage matches that shown on the identification plate (100-240 V~ 50/60Hz).

Connect the hand control (1)

Connect the power supply cable (2).

### IMPORTANT

For Type B Class I devices, and to avoid all risks of electric shock, the machine should only be connected to a power supply that has protective earthing.

To connect the power supply, only use the original cable supplied with the machine.

Check that the cables remain free around the device so that they do not get damaged.

Check that the machine is not damaged, in particular the protective housings.

## 11. Procedure to start the machine

Press the ON / OFF switch (3).

The display comes on, the machine carries out a self-test and the display shows in succession

Your device Kinetec Maestra™ is ready to be used.

**Warning:** Always check the motion parameters displayed on the hand control before starting the device.

## 12. Using the hand control

### 12.1. Changing the display language

Beginning	Keys to press	Display	Remarks
Switch the unit ON		KINETEC MAESTRA V01.0 KINETEC PLEASE WAIT KINETEC 40 STOP 50 110	Check that the hand control is not locked (See page 6).
Press simultaneously on the 2 keys	speed force	Language: ENGLISH	The display indicates the language selected.
To change the language	+ - Or	Language: FRENCH	The French language is selected. Available languages: English, French, German, Italian and Spanish.
To confirm the new language	limit	OK SWITCH ON/OFF	Switch the machine off and then on again to apply the changed display language.

## 12.2. START / STOP / REVERSE function

As with all KINETEC® systems, the device Kinetec Maestra™ is equipped with a START/STOP/REVERSE function.

Press the  key of the hand control, the movement stops,

Press the  key of the hand control, the movement starts in the opposite direction.

### CAUTION

For optimum safety, always give the hand control to the patient before starting the system.

## 12.3. Procedure to stop the machine

To stop the machine's movement: press the  key

To switch the unit off: press the ON / OFF switch (3) (see page 5)

## 12.4. Locking - Unlocking the hand control setting

The hand control allows the patient to control the machine as appropriate.

Simultaneously press the  &  keys to lock the hand control.

The display reads **LOCK**, you cannot change the parameters; if you try the message **LOCK** will be displayed. The Start and Stop keys are still active.

To unlock the hand control, simultaneously press the same keys. The display will show **UNLOCK**.

**We recommend that you lock the hand control when you give it to the patient.**

**Note:** the hand control lock is cancelled when you switch the unit ON/OFF.

## 12.5. Possible values for each parameter

									
		1 <sup>st</sup> press	2 <sup>nd</sup> press	3 <sup>rd</sup> press			1 <sup>st</sup> press	2 <sup>nd</sup> press	3 <sup>rd</sup> press
• Lower limit	-90°	-50°	-50°	-30°	0°	-30°	0°	0°	0°
• Upper limit	90°	90°	140°	60°	180°	225°	90°	180°	70°
• Speed	1 to 9 (from 150° to 440° per minute)								
• Force	1 to 6								
• Extension pause	0 to 900 seconds (15 minutes)								
• Flexion pause	0 to 900 seconds (15 minutes)								
• Timer	No timer (00H00) to 24H00								

### Default setting values for each parameter

									
		1 <sup>st</sup> press	2 <sup>nd</sup> press	3 <sup>rd</sup> press			1 <sup>st</sup> press	2 <sup>nd</sup> press	3 <sup>rd</sup> press
• Lower limit	-10°	-20°	0°	0°	0°	0°	0°	0°	0°
• Upper limit	10°	20°	60°	40°	90°	50°	40°	90°	40°
• Speed	2								
• Force	6								
• Extension pause	0								
• Flexion pause	0								
• Timer	0								

## 12.6. Quick Start




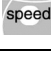
Set up the patient and proceed as below:

Beginning	Keys to press	Display	Remarks
Switch the unit ON		<b>KINETEC MAESTRA V01.4</b>	Check that the hand control is not locked (See page 6).
		<b>Please Wait</b>	
		<b>PRONO/SUPINATION -10 STOP 5 10</b>	
To choose the movement		<b>PRONO/SUPINATION -10 STOP 5 10</b>	The display shows the movement selected, with the default settings. Successive presses on the keys  or  allows you to select different movements.
		<b>WRIST EXT/FLX -20 STOP 96 20</b>	
		<b>WRIST+MP EXT/FLX 0 STOP 96 60</b>	
		<b>ULNAR DEVIAT. 0 STOP 96 40</b>	
		<b>THUMB OPPOSITION 0 STOP 96 90</b>	
		<b>LG FING. EXT/FLX 0 STOP 96 50</b>	
		<b>INTRINSIC + 0 STOP 96 40</b>	
<b>INTRINSIC - 0 STOP 96 90</b>			
<b>DISTAL EXT/FLX 0 STOP 96 40</b>			
To start the session with the original parameters for the movement (default setting).		<b>DISTAL EXT/FLX 0 RUN 20 40</b>	The angle display changes with the current movement.

**Note:** the values in the display column are given as an example. In actual use, they will come from stored programs or previous sessions.

**Warning:** Always check the motion parameters displayed on the hand control before starting the device.

## 12.7. To adjust the basic parameters of the movement

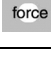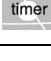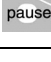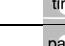
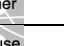
Beginning	Keys to press	Display	Remarks
To stop the unit		<b>PRONO/SUPINATION</b> -10 STOP 5 10	Check that the hand control is not locked (See page 6).
To choose the movement		<b>PRONO/SUPINATION</b> -10 STOP 5 10	The display shows the movement selected, with the default settings. Successive presses on the keys  or  allows you to select different movements.
		<b>WRIST EXT/FLX</b> -20 STOP 96 20	
		<b>WRIST+MP EXT/FLX</b> 0 STOP 96 60	
		<b>ULNAR DEVIAT.</b> 0 STOP 96 40	
		<b>THUMB OPPOSITION</b> 0 STOP 96 90	
		<b>LG FING. EXT/FLX</b> 0 STOP 96 50	
		<b>INTRINSIC +</b> 0 STOP 96 40	
		<b>INTRINSIC -</b> 0 STOP 96 90	
<b>DISTAL EXT/FLX</b> 0 STOP 96 40			
To display the upper or lower limit of the movement		<b>DISTAL EXT/FLX</b> 0 FLEX 96 <b>40</b>	The value blinks.
		<b>DISTAL EXT/FLX</b> <b>0</b> EXT 96 70	
To change the limit if necessary		<b>DISTAL EXT/FLX</b> <b>20</b> EXT 96 70	The new value blinks.
To confirm the new value, press another key or wait more than 3 seconds for automatic confirmation.		<b>DISTAL EXT/FLX</b> SPEED 2	While the value blinks press the  or  key to change if necessary.
		<b>DISTAL EXT/FLX</b> FORCE _	
		<b>DISTAL EXT/FLX</b> TIMER 00H00MIN	
Or		<b>DISTAL EXT/FLX</b> PAUSE HIGH 0s	Successive presses on this key selects the pause in the lower or upper limit.
To display the pause at the lower limit or upper limit		<b>DISTAL EXT/FLX</b> PAUSE LOW 0s	
To change the pause if necessary		<b>DISTAL EXT/FLX</b> PAUSE LOW <b>15s</b>	The new pause value blinks.
To confirm the new value press another key or wait more than 3 seconds. The display shows the selected mode		<b>DISTAL EXT/FLX</b> 0 STOP 94 70	The unit is ready to start with the new parameters.

**Note:** the values in the display column are given as an example. In actual use, they will come from stored programs or previous sessions.

**Warning:** Always check the motion parameters displayed on the hand control before starting the device.

## 12.8. PROGRAM MODE

### 12.8.1. To enter a program

Beginning	Keys to press	Display	Remarks
To switch off the unit			
Press the 2 keys at the same time and switch the unit ON	 	<b>KINETEC MAESTRA V01.4</b>	Welcome text displayed for 3 seconds
Then		<b>PROGRAM 2 EMPTY</b>	The program number blinks.
To change the program if necessary	 or 	<b>PROGRAM 10 EMPTY</b>	The new program number blinks.
To choose the movement		<b>PRONO/SUPINATION -10 PRG no. 10 10</b>	The display indicates the movement. Successive presses on the keys  or 
		<b>WRIST EXT/FLX -20 PRG no. 10 20</b>	
		<b>WRIST+MP EXT/FLX 0 PRG no. 10 60</b>	
		<b>ULNAR DEVIAT. 0 PRG no. 10 40</b>	
		<b>THUMB OPPOSITION 0 PRG no. 10 90</b>	
		<b>LG FING. EXT/FLX 0 PRG no. 10 50</b>	
		<b>INTRINSIC + 0 PRG no. 10 40</b>	
To display the extension or flexion limit of the movement		<b>INTRINSIC - 0 PRG no. 10 90</b>	allows you to select different movements.
		<b>DISTAL EXT/FLX 0 PRG no. 10 40</b>	
		<b>DIP EXT/FLEX 0 PRG no. 10 40</b>	
To change the limit if necessary	 or 	<b>DISTAL EXT/FLX 20 PRG no. 10 70</b>	The value blinks.
To confirm the new value, press another key or wait more than 3 seconds		<b>SPEED: 2</b>	While the value blinks press the  or  key to change if necessary.
		<b>FORCE: _</b>	
		<b>TIMER 00H00MIN</b>	
		<b>PAUSE LOW 0s</b>	
To record program 10		<b>Save: + Clear:-</b>	
Then		<b>PROGRAM 10 SAVING</b>	Program 10 has been recorded and the display indicates the next program so you can change another program.
OR		<b>PROGRAM 10 CLEARING</b>	Program 10 has been removed and the display indicates the next program so you can change another program.
To cancel the program			
To exit program mode, switch the unit OFF and switch back ON.		<b>KINETEC MAESTRA V01.0</b>	To use the modified program see page 10.

## 12.8.2. Using Programs

The device Kinetec Maestra™ allows you to store up to 16 programs (numbered 1 to 16), including the type of movement, ROM, speed, load, pauses and timer.

The original parameter values of the program are empty. These values can be modified and saved at any time (see 'To enter a program', page 9)

Beginning	Keys to press	Display	Remarks
To stop the unit		<b>PRONO/SUPINATION</b> -10 STOP 5 10	Check that the hand control is not locked (See page 6).
To access program mode		<b>WRIST EXT/FLX</b> -30 PRG <b>no.2</b> 30	The program number blinks.
To change the program if necessary	or	<b>THUMB OPPOSITION</b> 0 PRG <b>no.5</b> 30	The new program number blinks.
To exit and confirm the selected program		<b>THUMB OPPOSITION</b> 0 STOP 10 30	The current parameters are those recorded in program 5.
To exit without confirming the selected program		<b>PRONO/SUPINATION</b> -10 STOP 5 10	Back to the starting parameters.
To start the unit		<b>THUMB OPPOSITION</b> 0 RUN <b>20</b> 30	The angle display changes with the current movement.

**Warning:** Always check the motion parameters displayed on the hand control before starting the device.

## 12.8.3. Reading the values of a program: e.g. SPEED

Beginning	Keys to press	Display	Remarks
To stop the unit		<b>WRIST EXT/FLX</b> 25 STOP 20 90	Check that the hand control is not locked (See page 6).
To access program mode		<b>PRONO/SUPINATION</b> -10 PRG <b>no.4</b> 10	The program number blinks.
To change the program if necessary	or	<b>THUMB OPPOSITION</b> 0 PRG <b>no.3</b> 50	The new program number blinks.
To read the speed value		<b>THUMB OPPOSITION</b> SPEED: 2	Displays the speed value.
After 15 seconds or after pressing on another key		<b>WRIST EXT/FLX</b> 25 STOP 20 90	
To exit and confirm the selected program		<b>THUMB OPPOSITION</b> 0 STOP 20 50	The current parameters are those recorded in program 3.
To start the unit		<b>THUMB OPPOSITION</b> 0 RUN <b>20</b> 50	The angle display changes with the current movement.

### Comments:

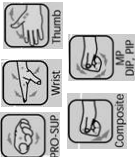
- The values shown in the 'Display' column are examples. They actually depend on the stored programs.
- The current movement parameters can be changed while using that program but no data will be stored in the original program. See Program Mode (page 9) to modify programs.

### Comments:

- When a program has been deleted, the display shows

PROGRAM 11  
EMPTY

## 12.8.4. Program table:

• Program number	Movement type	Lower limit	Upper limit	Speed	Load	Pause at upper limit	Pause at lower limit	Timer
1								
2								
3								
4								
5								
6								
7								
8								
9								
10								
11								
12								
13								
14								
15								
16								

## 12.9. To define the upper and lower movement limits

### 12.9.1. At the start of a session

This function, MANUAL MODE, allows the maximum limits that a patient can tolerate to be defined and recorded at the beginning of a session.

Set up the patient and proceed as below:

Beginning	Keys to press	Display	Remarks
Switch the unit ON		<b>KINETEC MAESTRA V01.0</b> <b>KINETEC PLEASE WAIT</b> <b>PRONO/SUPINATION -10 STOP 5 10</b>	Check that the hand control is not locked (See page 6).
To select MANUAL MODE	continuously press 	<b>PRONO/SUPINATION -10 MANUAL 15 10</b>	The unit starts moving to the upper limit of the movement. The settings are controlled by you.
To set the pain level when reached, immediately press	 limit	<b>PRONO/SUPINATION -10 MANUAL 15 15</b>	The new upper value limit of the movement is recorded.
To set the lower limit	continuously press 	<b>PRONO/SUPINATION -10 MANUAL 5 15</b>	The unit starts moving to the lower limit of the movement.
To set the pain level when reached, immediately press	 limit	<b>PRONO/SUPINATION -5 MANUAL 5 15</b>	The new lower value limit of the movement is recorded.
To start the session with the new movement limits		<b>PRONO/SUPINATION -5 RUN 8 15</b>	The angle display changes with the current movement.

### 12.9.2. During the session

This function, BYPASS MODE, allows the maximum limits that a patient can tolerate to be defined and recorded, which allows you to work on increasing amplitude.

Beginning	Keys to press	Display	Remarks
The unit is running		<b>PRONO/SUPINATION -5 RUN 8 15</b>	Check that the hand control is not locked (See page 6).
To select BYPASS Mode	continuously press 	<b>PRONO/SUPINATION -5 BYPASS 40 15</b>	The unit exceeds the recorded upper limit.
To set the new pain level when reached, immediately press	 limit	<b>PRONO/SUPINATION -5 BYPASS 40 40</b>	The new upper value limit of the movement is recorded.
To set the lower limit	continuously press 	<b>PRONO/SUPINATION -10 BYPASS 30 40</b>	The unit starts moving to the lower limit of the movement.
To set the new pain level when reached, immediately press	 limit	<b>PRONO/SUPINATION -30 BYPASS 30 40</b>	The new lower value limit of the movement is recorded.
Continue the session with the NEW movement limits.		<b>PRONO/SUPINATION -30 RUN 20 40</b>	The angle display changes with the current movement.

#### Comments:

- The values shown in the 'Display' column are examples.

## 13. Mechanical adjustments

### 13.1. Setting up the forearm support

The forearm support should be on the RIGHT side of the machine for the LEFT Hand, and on the LEFT side for the RIGHT Hand.  
For prono-supination or thumb opposition, place the forearm support IN FRONT of the motor axis (see page 15 or 17).

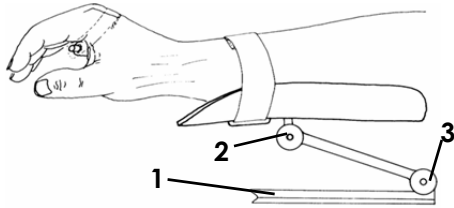
The forearm support is mounted on a telescopic rod (1). There are also two notched joints (2) and (3), which allow the wrist to be positioned:

- in extension,
- in ZERO position,
- or in flexion.


Position for the left hand


Extension

Zero Position

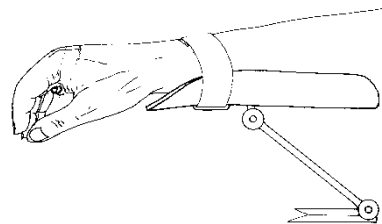Flexion

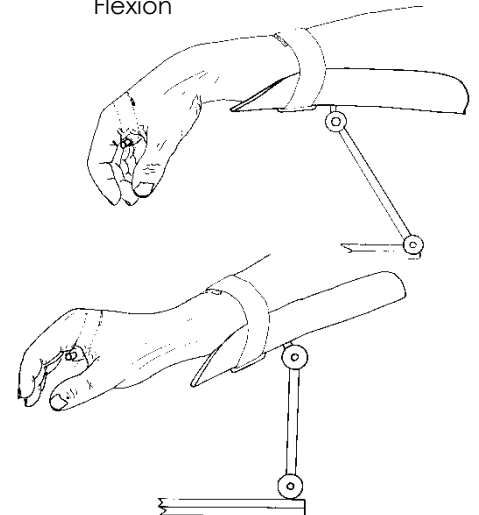
The joints also permit the forearm support to be tilted up or down. However, if this is done, the angle of tilt should be taken into account when setting the range of motion on the display (see page 6).


### 13.2. Setting up the driving mechanism

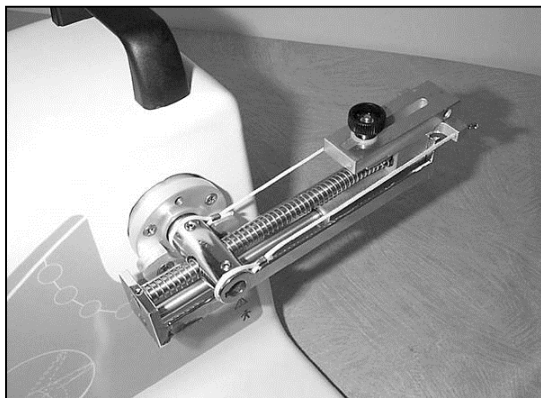The driving mechanism should be attached to the motor axis.  
In order to facilitate its setting up, use the intrinsic + movement and stop the unit at 0°.

LEFT hand


RIGHT hand
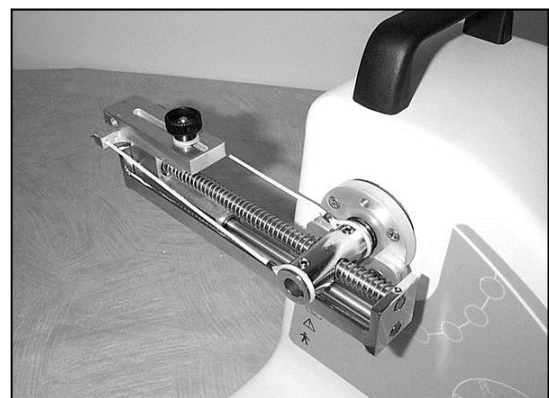
#### Note


Make sure that when the driving mechanism is inserted on the motor axis, the cable (4) is ABOVE the groove of the cam (5).

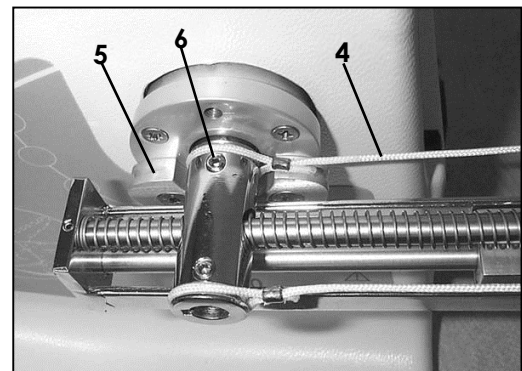The driving mechanism should then be locked in place by tightening the screw (6) with the Allen key included in the accessory case.



### 13.3. Adjusting the finger length

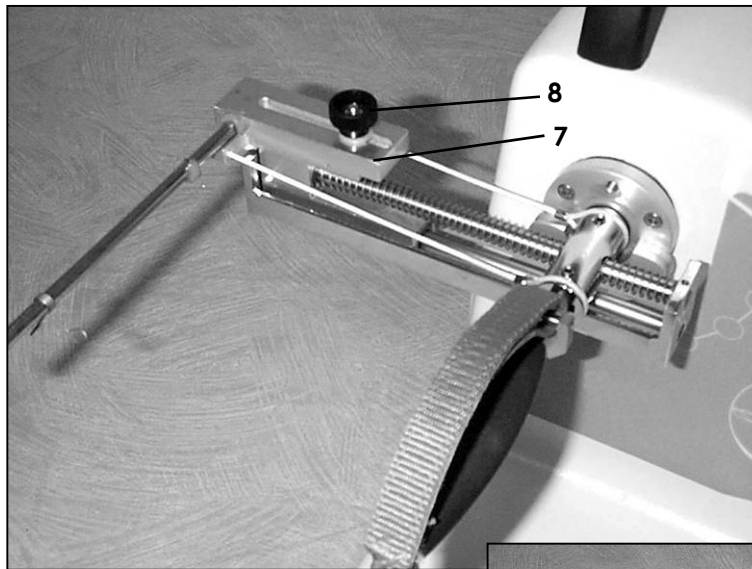In order to adapt the position of the mobilisation accessories to individual hand and finger lengths, the practitioner will orient the guide (7), in position (A) or in position (B). This is done by fully unscrewing the knurled nut (8) and lifting the guide (7).

Guide in position A

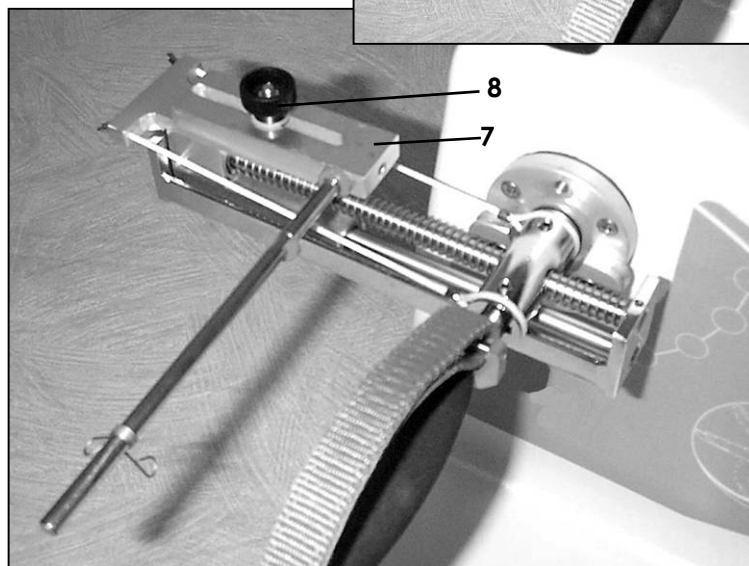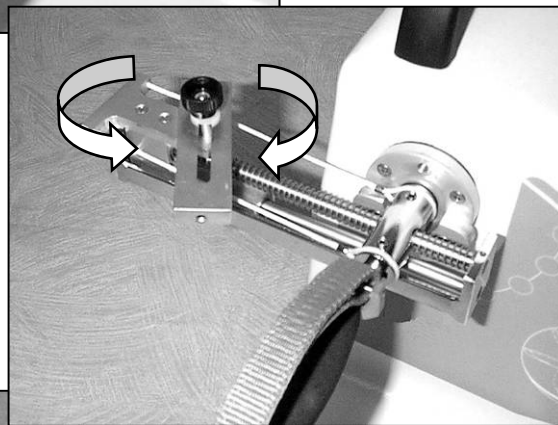Guide in position B

## 14. Possible movements

### 14.1. Pronation / Supination

The device Kinetec Maestra™ provides movements from 90° of pronation to 90° of supination.

- To select the Pronation / Supination movement:

- Press  and set the movement parameters (see page 8).
- or select a program  (see page 10).

### 14.2. Extension / Flexion of the wrist

The device Kinetec Maestra™ provides movements from -50° from 90°.

- To select wrist movement with extension/flexion:

- Press  and set the movement parameters (see page 8).
- or select a program  (see page 10).

### 14.3. Extension / Flexion of the wrist and MP joints of the fingers

The device Kinetec Maestra™ provides movements from -50° from 140°.

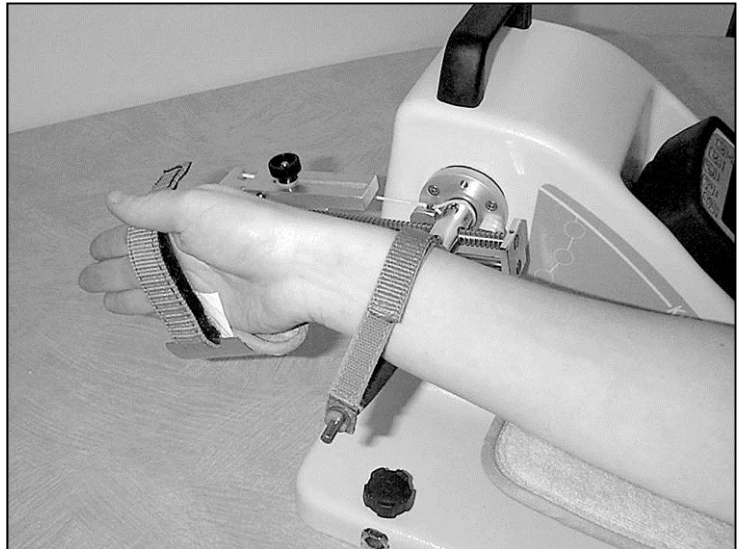
- To select wrist movement with extension / flexion + MP:

- Press  2 times and set the movement parameters (see page 8).

- or select a program  (see page 10).

### 14.4. Ulnar deviation

The device Kinetec Maestra™ provides movements from -30° from 60°.


- To select wrist movement with ulnar deviation:

- Press  3 times and set the movement parameters (see page 8).


- or select a program  (see page 10).

## 14.5. Thumb opposition

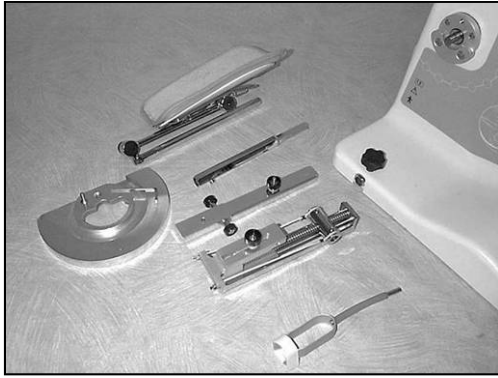
The device Kinetec Maestra™ provides movements from 0° from 180°.


- Select the accessories

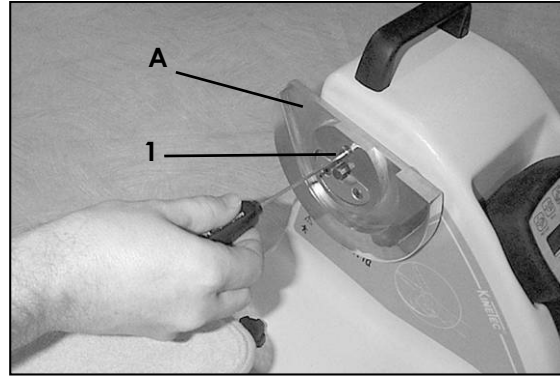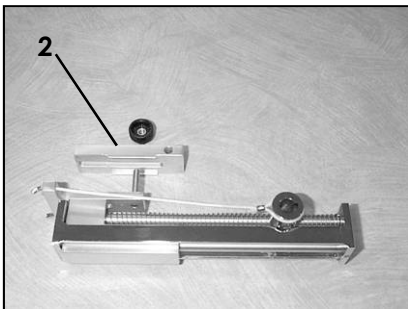
- Set up the guide disc for the thumb movement  
 - Make sure that the opening (A) is correctly positioned.  
 - Fix the guide in position with the screw (1).

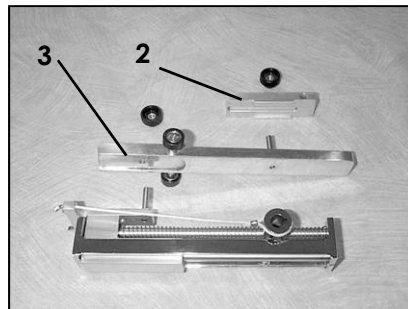
- Remove the guide (2) from the driving mechanism.

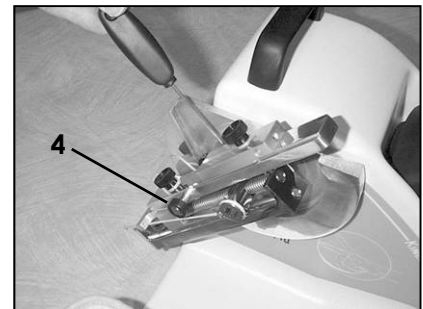
- Set up the driving mechanism extension (3).  
 - Install the guide (2).

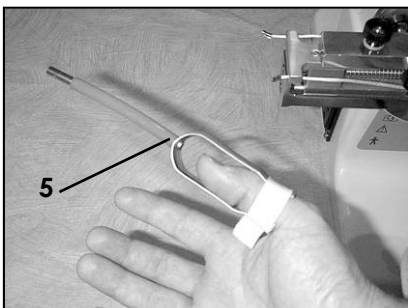- Install the driving mechanism (4) on the motor axis.  
 (see p. 14 for more details)

- Set up the thumb attachment (5)

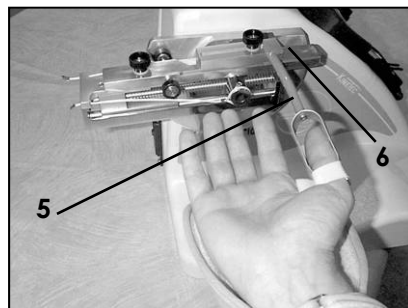- Fix the thumb attachment (5) on the driving mechanism.  
 - Tighten the screw (6).


• To select thumb opposition movement:

- Press  and set the movement parameters (see page 8).

- or select a program  (see page 10).

## 14.6. Extension / Flexion of the 3 joints of the long fingers

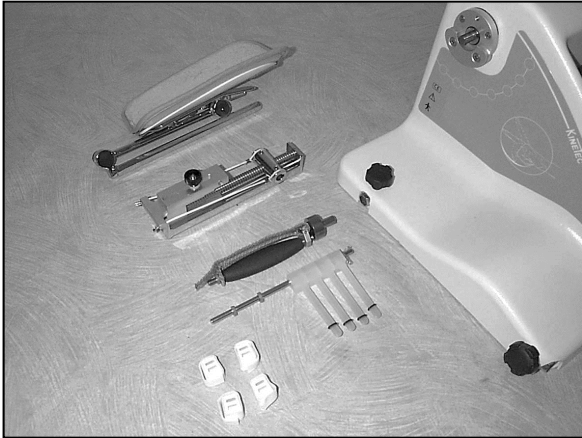The device Kinetec Maestra™ provides movements from -30° from 225°.

- To select Extension / Flexion of the 3 joints of the long fingers:
  - Press  and set the movement parameters (see page 8).
  - or select a program  (see page 10).

## 14.7. Intrinsic +

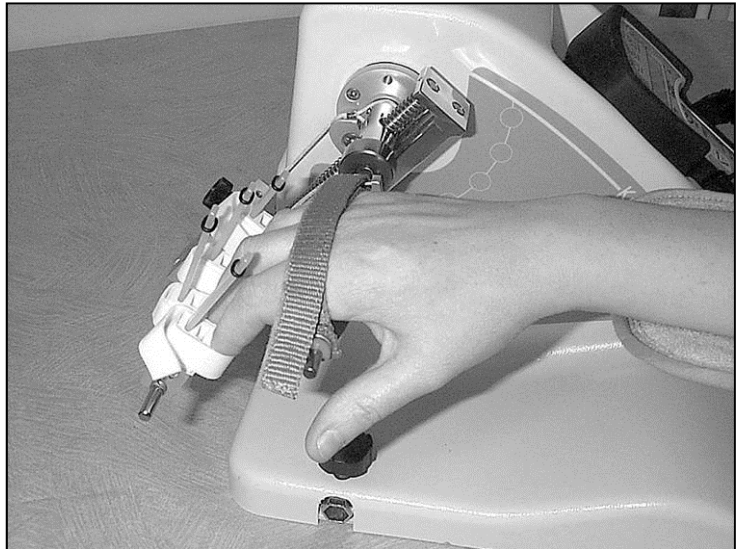
The device Kinetec Maestra™ provides movements from 0° from 90°.

- To select the intrinsic + movement:
  - Press  and set the movement parameters (see page 8).
  - or select a program  (see page 10).

## 14.8. Intrinsic -

The device Kinetec Maestra™ provides movements from 0° from 180°.


- To select the intrinsic - movement:

- Press  and set the movement parameters (see page 8).
- or select a program  (see page 10).

## 14.9. Extension / Flexion of the DIP joint

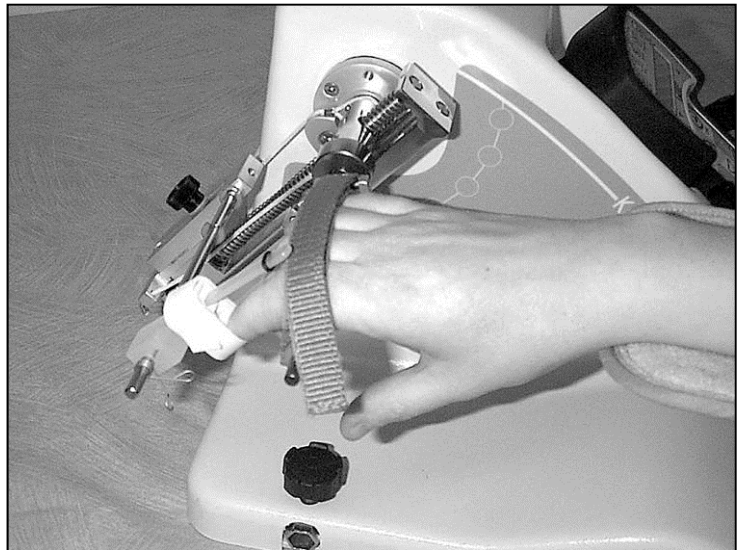The device Kinetec Maestra™ provides movements from 0° from 70°.

- To select Extension / Flexion of the DIP joint:

- Press  and set the movement parameters (see page 8).
- or select a program  (see page 10).

## 15. Product information

### 15.1. Maintenance

After 2,000 hours of operation, or once a year, the device Kinetec Maestra™ requires lubrication and maintenance operations (lubrication of the joints and pointer stops). When the machine is switched on the message **SERVICE TIME Mx** is displayed to indicate that the service interval has been reached. Despite this indication, you can continue to use your machine by pressing **[START]**, but you should contact your nearest KINETEC® technician to have the maintenance operations carried out as soon as possible. An after-sales service inspection sheet and the technical catalogue are available on request from your KINETEC® distributor.

**WARNING:** Before using this machine, always check that the electrical socket is in good condition and is suitable for the splint power supply cord. Only use the original cable supplied with the machine. Check that the cables remain free around the device so that they do not get damaged.

**WARNING:** Before using this machine, always check that the machine is not damaged, in particular the protective housings.

When the machine is no longer in working condition, please return it to us, together with its accessories, for destruction.

A motor running time counter is available by simultaneously pressing keys  and  ;

the displays shows **RESET TIME 215H** (this is an example).

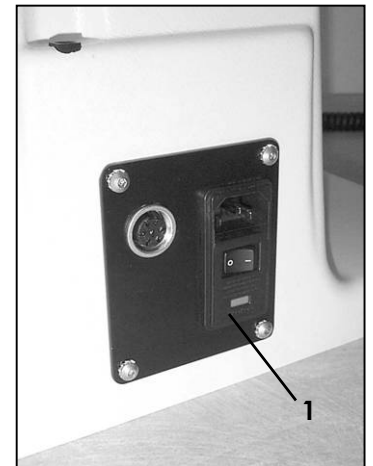
This counter can be reset by pressing the key .

### 15.2. Troubleshooting guide

A spare parts list and technical catalogue are available on request from your KINETEC® distributor.

If, after connecting the power supply cable to the power supply and switching on the machine:

- The display does not indicate any information:
    - Check that the electrical socket is live using another device or voltmeter.
    - Replace the fuse(s) (1) of the connector with fuses of the same type and calibre: 2 fuses T 750 mA 250V (6.3 x 32) (KINETEC® order: 4610007434).
    - Check that the hand control is connected properly.
    - If the display still does not indicate any information, contact your nearest KINETEC® technician.
  - Your machine does not work and the display indicates **50 STOP 25 115**, press **[START]** again.
- If your machine still does not work, contact your nearest KINETEC® technician.
- Your machine does not work and the display indicates:
    - "SERVICE D1": angle measurement function failure,
    - or "SERVICE D2": no movement,
    - or "SERVICE D3": abnormal electrical consumption by motor,
    - or "PUSH STOP/START": power failure or disconnected motor.switch the machine off then on,  
contact your nearest KINETEC® technician if the same message is displayed.

### 15.3. Cleaning

Before carrying out any cleaning operation, SWITCH OFF the unit and disconnect the power supply. In order to ensure optimal hygiene, you are advised to clean the machine for each new patient.

Cleaning should be carried out in the environmental conditions specified in the "Technical Specifications" section below.

Use a DISINFECTANT product (alcohol-free or <5% alcohol solution) in spray (plastic cases and metal components).

In order to ensure optimal hygiene, you are advised to clean the covers for each new patient. All the consumables enable hazard-free disposal.

### 15.4. Disposal and recycling

- a • Packaging:** The packaging must be separated into plastic and paper / cardboard components and taken to special recycling sites.
- b • KINETEC® patient pad kit:** Clean with a disinfectant product then take it to special recycling sites.
- c • Unit:** It contains electronic components, cables, aluminium, steel and plastic parts. When the machine is no longer operational, disassemble it, separate it into different types of material and take these to authorised recycling centres or return the machine to Kinetec SAS for destruction. Or contact the local authorities to determine the appropriate method of disposal for parts and accessories that are potentially hazardous to the environment.

## 15.5. Technical specifications

### Product:

Lifespan of the machine:	12 years
Weight:	8Kg (17.6 pounds) (without case)
Splint dimensions:	35cm (14 inches) x 41cm (16 inches) x 30cm (12 inches)
Angular limits:	see pages 15 to 19
Speeds:	from 150 to 440° per minute
Patient sizes:	all hand sizes from 7 years old
Maximum weight of the user:	135 kg (297 pounds)
Acoustic pressure:	<70dB
Applied parts:	Remote control, hygienic pads and/or comfort support.

### Electricity:

Power supply:	100-240V~
Frequency:	50-60 Hz
Power consumption:	50 VA
Class:	Device of Type BF Class II or Type B Class I
Protection class (device):	IP 20 (protected against solid objects greater than 12.5mm, but not protected against liquids)
Protection class (carrying case):	IP 01 (non-protected against solid foreign objects, protected against vertically falling water drops)
Fuse:	T 750mA 250V 6.3 x 32mm (KINETEC® order: 4610007434)

### Environment:

Storage/transport conditions:	Temperature:	-25 to 70°C / -13 to 158°F.
	Relative humidity:	up to 93% without condensation.
Operating conditions:	Temperature:	5 to 40°C / 41 to 104°F.
	Relative humidity:	15% to 93% without condensation.
	Atmospheric pressure:	700 hPa to 1060 hPa.

## 15.6. Symbols used

	CAUTION (consult the accompanying documentation)		Speed		Flexion limit
	STOP (power off)		Timer		Pronation / supination movement
	ON (power on)		Force		Wrist movement
	Start movement		Pause		Thumb movement
	Stop movement		Increase		Long fingers complete movement key
	Program access		Decrease		Selective movement
	Right way up when box is stored		Extension limit		Alternating current
	Contains electric and electronic components; do not throw away with household refuse.		Fragile		Keep dry during storage or transport
	Temperature Limit during storage or transport		Follow the instructions for use		Follow the instructions for use
	Class II device		TYPE BF device (protection against electric shocks)		TYPE B device (protection against electric shocks)
IP20 IP01	See "Protection class" in section "Technical specifications"		Humidity limit during storage or transport		

## 15.7. Warranty

The KINETEC® warranty is strictly limited to the replacement, free of charge, or to factory repairs of part(s) recognised as defective.

Kinetec SAS guarantees its continuous passive motion systems for 2 years against all defects of manufacture from the date of purchase by the consumer.

Kinetec SAS is the only organization able to assess the application of the warranty to its systems.

The warranty will be considered null and void if the device has been used abnormally or under conditions of use other than those indicated in the user's manual.

The warranty will also be considered null and void in the event of deterioration or an accident due to negligence, inappropriate surveillance or inappropriate maintenance, or due to transformation of the equipment or an attempt to repair the equipment.

## 15.8. Guidance and manufacturer's declaration

<b>Guidance and manufacturer's declaration - Electromagnetic emissions</b>			
The device Kinetec Maestra™ is intended for use in the electromagnetic environment specified below. The customer or the user of the device Kinetec Maestra™ should ensure that it is used in such an environment.			
Emissions test	Compliance	Electromagnetic environment – guidance	
Radio frequency emissions CISPR 11	Group 1	The device Kinetec Maestra™ uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
Radio frequency emissions - CISPR 11	Class B	The device Kinetec Maestra™ is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.	
Harmonic emissions - IEC 61000-3-2	Class A		
Voltage fluctuations / Flicker emissions - IEC 61000-3-3	Complies		
<b>Guidance and manufacturer's declaration - Electromagnetic immunity</b>			
The device Kinetec Maestra™ is intended for use in the electromagnetic environment specified below. The customer or the user of the device Kinetec Maestra™ should ensure that it is used in such an environment.			
Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment – Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±2 kV, ±4 kV, ±6 kV contact	±2 kV, ±4 kV, ±6 kV contact	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
	±2 kV, ±4 kV, ±8 kV air	±2 kV, ±4 kV, ±8 kV air	
Electrical fast transient / burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0.5 kV, ±1 kV between lines	±0.5 kV, ±1 kV between lines	Mains power quality should be that of a typical commercial or hospital environment.
	±0.5 kV, ±1 kV, ±2 kV between line and earth	±0.5 kV, ±1 kV, ±2 kV between line and earth	
Voltage interruptions IEC 61000-4-11	< 5% $U_T$ (>95% dip in $U_T$ ) for 5 seconds	< 5% $U_T$ (>95% dip in $U_T$ ) for 5 seconds	When the interruption occurs, the device Kinetec Maestra™ is reset. After turning on, push START to begin the session.
Voltage dips and voltage variations on power supply input lines IEC 61000-4-11	< 5% $U_T$ (>95% dip in $U_T$ ) for 0.5 cycle	< 5% $U_T$ (>95% dip in $U_T$ ) for 0.5 cycle	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device Kinetec Maestra™ requires continued operation during power supply interruptions, we recommend powering the device Kinetec Maestra™ using an uninterruptible power supply or a battery.
	40% $U_T$ (60% dip in $U_T$ ) for 5 cycles	40% $U_T$ (60% dip in $U_T$ ) for 5 cycles	
	70% $U_T$ (30% dip in $U_T$ ) for 25 cycles	70% $U_T$ (30% dip in $U_T$ ) for 25 cycles	
Power frequency (50/60 Hz) magnetic field - IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: $U_T$ is the AC mains voltage prior to application of the test level.			

**Guidance and manufacturer's declaration - Electromagnetic immunity**

The device Kinetec Maestra™ is intended for use in the electromagnetic environment specified below. The customer or the user of the device Kinetec Maestra™ should ensure that it is used in such an environment.

Immunity test	Test level according to IEC 60601	Compliance level	Electromagnetic environment – Guidance
Conducted RF interference IEC 61000-4-6	3 V <sub>eff</sub> from 150 kHz to 80 MHz	3 V	<p>Mobile and portable RF communication devices should not be used closer to any part of the device Kinetec Maestra™ machine, including its cables, than the recommended separation distance, calculated based on the equation applicable to the emitter's frequency.</p> <p><b>Recommended separation distance</b></p> $d = 1.2 \sqrt{P}$ <p><math>d = 1.2 \sqrt{P}</math> from 80 MHz to 800 MHz</p> <p><math>d = 2.3 \sqrt{P}</math> from 800 MHz to 2.5 GHz</p> <p>where <math>P</math> is the emitter's maximum output power characteristic in watts (W), according to the emitter's manufacturer, and <math>d</math> is the recommended separation distance in metres (m).</p> <p>The field intensities of fixed RF emitters, determined by an on-site electromagnetic investigation<sup>a</sup>, should be below the compliance level in each frequency range<sup>b</sup>.</p> <p>There may be interference near appliances bearing the following symbol:</p> 
Radiated RF interference IEC 61000-4-3	3 V/m from 80 MHz to 2.5 GHz	3 V/m	

NOTE 1 At 80 and 800 MHz, the highest frequency range is applicable.

NOTE 2 These directives cannot be applied in every situation. Electromagnetic propagation is affected by absorption and reflection by structures, objects and people.

- <sup>a</sup> The field intensity of fixed emitters such as base stations for radio-telephones (cellular/cordless) and land mobile radios, amateur radio, AM/FM radio broadcasts and TV broadcasts cannot be predicted exactly in theory. To evaluate the electromagnetic environment due to fixed RF emitters, an on-site electromagnetic investigation should be considered. If the field intensity measured where the device Kinetec Maestra™ machine is used exceeds the aforementioned applicable RF compliance level, the device Kinetec Maestra™ machine should be monitored to check that it is working normally. If abnormal results are observed, additional measures may be necessary, such as reorienting or repositioning the device Kinetec Maestra™.
- <sup>b</sup> Over the frequency range 150 kHz to 80MHz, field intensities should be less than 3V/m.

**Recommended separation distances between mobile and portable RF communication devices and the device Kinetec Maestra™ machine**

The device Kinetec Maestra™ machine is designed to be used in an electromagnetic environment in which radiated RF interference is controlled. The customer or user of the device Kinetec Maestra™ machine can help prevent electromagnetic interference by maintaining a minimum distance between mobile and portable RF communication devices (emitters) and the device Kinetec Maestra™ machine, as recommended below, according to the communication device's maximum output power.

Maximum assigned output power for the emitter W	Separation distance according to the emitter's frequency m		
	from 150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	from 800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For emitters whose assigned maximum emitted power is not given above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the emitter frequency, where  $P$  is the emitter's maximum emission power characteristic in watts (W), according to the latter's manufacturer.

NOTE 1 At 80 and 800 MHz, the separation distance for the highest frequency range is applicable.

NOTE 2 These directives cannot be applied in every situation. Electromagnetic propagation is affected by absorption and reflection by structures, objects and people.

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