Biofeedback & EMG-getriggerte Stimulation
Biofeedback & EMG-triggered stimulation

DuoBravo

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Content

Please read the Operators Manual carefully
before using the device and pay attention
to all references !

In the following you'll find all information directly necessary for performing the treatment
with grey background.

	Page
I. Biofeedback- / ETS-device DuoBravo	
Delivery & Accessories	54
Technical Data	55
Explanation of Symbols	56
Safety Adjustments	56
Introduction – What is DuoBravo ?	57
Indication	58
Contraindication	59
Warnings	60
Advice on Application of Electrodes	61
Maintenance	62
Warranty	62
II. Display and Operating Elements	
Operating Elements	63
Battery Compartment	63
Connections	64
The Display	65
The Info-Line	66
III. Preparation of DuoBravo	
Basic Settings	67
Setup Real Time Clock	68
Setup Language	68
Preparation of Therapy	69

Content

	Page
IV. Therapy with DuoBravo	
The Different Phases	70
Use of pre-set programs	71
Therapy in a phase with pure stimulation	72
Therapy in an EMG-phase	73
Therapy in an ETS-phase	74
Therapy in combined programs	74
V. Additional Options for Configuration of DuoBravo	
Configuration of Free Programs	75
Parameters of Free Programs	76-79
Programming PC1	76
Programming PC2	77
Programming PC3	78
Programming PC4	79
Lock-Button	80
Threshold	80-81
Setting of threshold before start of treatment	80
Automatic adjustment of threshold during treatment	81
VI. Therapy-Control	
Therapy-Control	83
Statistics	83-84
VII. Programs	
Programs Urology	85-88
Programs Neurology	89-92
Programs Urology + Neurology	93-99
Anatomy	100
Configuration	101
Waste Disposal / Recycling of Batteries / Literature / Copyright	102
Electromagnetic Compatibility	106-108
Graphics	109

Delivery & Accessories

Included in the delivery:

REF: 010E-089: 1 unit **DuoBravo U** (Urology), 2 cables to connect the electrodes, 1 reference cable, 8 self-adhesive electrodes (40 x 40 mm), 1 stand, 4 batteries type AA and 1 operators manual

REF: 010E-090: 1 unit **DuoBravo N** (Neurology), 2 cables to connect the electrodes, 1 reference cable, 8 self-adhesive electrodes (40 x 40 mm), 1 stand, 4 batteries type AA and 1 operators manual

REF: 010E-093: 1 unit **DuoBravo U+N** (Urology + Neurology), 2 cables to connect the electrodes, 1 reference cable, 8 self-adhesive electrodes (40 x 40 mm), 1 stand, 4 batteries type AA and 1 operators manual

Accessories:

REF

011E-006: 1 unit self adhesive electrodes 40x40 (4 pieces)

011E-007: 1 unit self adhesive electrodes 40x80 (4 pieces)

011E-008: 1 vaginal probe “IncoProbe V”

011E-009: 1 rectal probe “IncoProbe R”

011F-010: 1 vaginal probe “Periform”

011F-011: 1 rectal probe “Anuform”

011F-012: 2 cables to connect the electrodes

011F-037: 1 tube electrode jelly for probes

011F-086: 1 storage box

011F-083: 1 reference cable

011E-095: 1 software for therapists

011F-096: 4 batteries type AA

011E-098: 1 operators manual **DuoBravo**

011E-099: 1 fiber optic cable

The battery life is approximately 12 hours or more at 20 mA per channel. The batteries can be stored for about 2 years. The service life (expiry date) of electrodes, probes and electrode jelly is printed on each package.

DuoBravo shall only be used with original accessories.

The expected service life of the **DuoBravo** is 5 years.

Technical Data

1. EMG

- 1.0 Dual Channel
- 1.1 EMG Range: 0.2 to 2000 μ V RMS (continuous)
- 1.2 Sensitivity: 0.1 μ V RMS
- 1.3 Selectable Bandpass filter - 3db Bandwidth,
 - a. Wide: 18 Hz +/- 4 Hz to 370 Hz +/- 10% - Reading below 235 microvolts
10 Hz +/- 3 Hz to 370 Hz +/- 10% - Reading above 235 microvolts
 - b. Narrow: 100 Hz +/- 5 Hz to 370 Hz +/- 10%
- 1.4 Notch filter: 50 Hz - 33 dbs
- 1.5 Common Mode Rejection Ratio: 130 dbs Minimum @ 50 Hz
- 1.6 Batteries: 4 x AA
- 1.7 Work / Rest periods: 2-99 seconds
- 1.8 Number of Trials: 2-99

2. Neuromuscular Stimulation

- 2.1 Dual Channel
- 2.2 0 - 90 mA into 500 ohm load, 70 mA maximum into 1000 Ohm load, 46 mA maximum into 1500 Ohm load. (Indication only, actual intensity varies depending on condition of skin electrodes).
- 2.3 Type: Constant current
- 2.4 Waveform: Symmetrical, rectangular, bi-phasic with net zero DC current
- 2.5 Pulse width selection: 50-450 μ S
- 2.6 Pulse Rate Selection: 2-100 Hz
- 2.7 Work / Rest periods: 0,5-99 seconds
- 2.8 Time 1 - 99 minutes
- 2.9 Ramp up time: 0.1 - 9.9 seconds
- 2.10 Preset and user programmable treatment programs
- 2.11 Automatic output shut off with detection of open electrode (> 0,5 mA)

Environmental conditions for storage & transport: 0-93% atmospheric humidity, -25 to +70 degree centigrade. Environmental conditions for use: 0-93% atmospheric humidity, +5 to +40 degree centigrade, air pressure 700-1060 hPa.

Dimensions: L 141,6 mm / W 74 mm / H 40,4 mm - Weight of the device: 275 g (incl. batteries)
Firmware version 3.0

The **DuoBravo** is protected against debris larger than 12.5 mm in diameter from entering the cabinet and is not water resistant and rated IP20.

Radio Link:

RF Frequency Range: 2.4000 GHz to 2.4835 GHz

RF Default Output Power: \leq 1,0 mW

RF Maximum Output Power: \leq 2,52 mW

Antenna Gain: 0 dBi

The graphics of the waveform and of the intensity depending on resistance can be found on page 109.

Explanation of Symbols

Attention
Warning

Type BF applied parts

Serial-Number
of the product

Keep dry

Batch code

Manufacturer

2012 - 08

Date of
manufacture

CE-mark: conforms to
essential requirements of
the MDD 93/42/EEC

Not suited for persons
with cardiac pacemaker

Read the instructions

Safety Adjustments

1. The +buttons become inactive after one minute running time of a program to avoid an unintentional increase of intensity. This lock is switched off again, only by pressing the +button three-times repeatedly. Now an increasing of intensity is possible again for another minute.
2. If the electrical circuit should be interrupted during operation (> 2 mA), for example by losing an electrode or by a broken lead wire, the device stops stimulating immediately.
3. If any phase comes to its end, the intensity falls down to 8 mA automatically and must be reset by the user to the desired setting.
4. The **DuoBravo** features a stimulation circuit failure detection system incorporated into the device and its software. Four pulses with a pulse width and voltage not exceeding 1 μ S and 6 volts are triggered before each main pulse to check the integrity of the circuit. These pulses cannot be felt and do not effect the efficacy of the device. If a failure is detected during this test, STIM F will display on the LCD. If STIM F is repeatedly displayed on the LCD then the unit should be returned to the manufacturer for testing. This is an added safety protection system to ensure each pulse generated by the device is safe for the user.

Introduction – What is DuoBravo ?

The **DuoBravo** is a modern device for biofeedback and neuromuscular stimulation. The term biofeedback refers to a method to perceive biological processes with the help of a technical device. This instrument measures physical activity - even if there are no visible or no voluntary muscle contractions - and converts them to optical and/or sound signals. Nevertheless **DuoBravo** is a combined two-channel device that can be flexibly used for electromyography (EMG) or EMG-triggered stimulation (ETS) or for neuromuscular stimulation only. Doctors and patients are the intended operators for this unit.

The **DuoBravo** is available in three different versions. **DuoBravo U** can be used for urologic, gynecologic and proctologic therapy while **DuoBravo N** is designed for neurologic and orthopedic treatments. The combination device **DuoBravo U + N** contains all programs. Each version includes preset EMG- and ETS-programs as well as multiple stimulation programs in different modes. Additionally the devices have 4 freely customizable programs in which individual STIM-, EMG- and/or ETS-phases and a combination can be set.

The clip on the back of the unit allows the user to fasten the device to the clothes, the unit is intended to be supported by hand during normal use. The belt clip is for temporary relief only. During treatment it's not necessary to hold **DuoBravo**, since it can stand upright on a flat surface in its stand. This allows the patient to train or to be treated while observing the success of the individual efforts. The muscle activity is indicated by a yellow-green bargraph on each side of the screen and by numerical values in the bottom-line of the display.

At the end of each session, the screen automatically displays the average values of muscle activity in this session. The **DuoBravo** can be used with or without connection to a personal computer. The stored treatment data can be transmitted to the PC at a later date using a software available separately.

This device is quality controlled and fulfills the rules and regulations set by the German law for medical products (MPG) as well as the guidelines of the European Community 93/42/EWG for medical products. Therefore this device carries the CE-mark „CE 0123“.

You can find the serial number of this device on the belt clip.

Manufacturer: MTR+ Vertriebs GmbH, Kamenzer Damm 78, 12249 Berlin,
www.mtrplus.com

Indications

According to the specific advices of the treating physician **DuoBravo** can be used without any help to support muscle training in the following areas.

Main indication for DuoBravo U / DuoBravo U+N:

- Elimination of light to mid strength faecal incontinence
- Elimination of light to mid strength urinary incontinence
 - Urge-incontinence
 - Stress-incontinence
 - Mixed-incontinence

Main indication for DuoBravo N / DuoBravo U+N:

- Stroke Rehabilitation
- Paresis
- EMG-Biofeedback Training
- Relaxation

Side indication for DuoBravo U / DuoBravo N / DuoBravo U+N:

- Improvement of blood flow
- Improvement of muscle spasticity
- Muscle Strengthening
 - Avoidance of muscular imbalances
 - Activation of muscular fibers
 - Muscle Fatigue
 - Strengthening of atrophied muscles
 - Muscle Recovery
- Conservation and extension of mobility
- Release of bodily endorphins
- General Pain Relief and Pain Therapy including but not limited to:
 - Arthritis
 - Back Pain including but not limited to:
 - Chronic Back Pain
 - Cervical Spine Pain
 - Thoracic spine Pain
 - Lumbar spine Pain
 - Circulation problems
 - Fracture Pain
 - Myalgia
 - Neuralgia
 - Phantom Pain

Contraindications

DuoBravo may under no circumstances be used in the following cases:

- By patients with electronic implants like pumps or a pace maker (as functional disturbances can occur)
- By patients with myocard damages (damage to the cardiac muscle) or arrhythmia (inconsistent heart beat)
- In presence of any flammable gas or air-gas mixture.
- When operating machinery with a potential of danger
- By patients with epilepsy
- If an inflammation of the vagina, anus or urinary tract is present.
- By patients with mechanical ileus
- By patients with febrile illnesses or infectious diseases

Please pay unconditional attention to the following fundamental rules:

- Do not stimulate transcerebraly (via the head).
- Do not stimulate via the Carotis-Sinus-Nerve (carotid artery).
- Do not stimulate in the area of the Sinus knot (heart).
- Do not stimulate on the Bulbous (eye).
- Do not stimulate in the Larynx-Pharynx-Area (throat / mouth).
- Do not stimulate in places with skin defects.
- Do not position the electrodes directly on the spine.

Take special care with the use of the electric stimulation:

- if stimulation is located close to osteosynthesis material (metal in the body).
- if after an operation the healing process can be interrupted through a muscle contraction.
- after acute bleeding or bone fractures with a tendency to bleed.
- in case of menstruation or pregnancy.
- in case of de-sensitization of the skin (damaged nerves).
- if patients cannot use the device properly, due to mental or physical limitations.

Warnings

- **DuoBravo** is a type BF equipment and may not be immersed in liquids.
- If using rechargeable batteries be sure to use a CE approved battery charger. Never connect the **DuoBravo** directly to the mains, to a battery charger or to any other net-powered equipment.
- The operator may not touch the battery contacts & patient simultaneously.
- To avoid the effects of electromagnetic interference never use the **DuoBravo** in the EMG-mode, with less than 4 metres distance to a cellular telephone or near any other powerful radio interference producing equipment that causes electrical signals etc. In the EMG Mode, the **DuoBravo** may be susceptible to strong interfering radio type emissions that may lead to temporally increased EMG microvolt readings. If suspected interference is causing a problem with EMG readings, change position of the device and move away from any electrical emitting equipment. The reading will immediately return to the correct value when the interference ceases.
- The power supply of some laptop computers may cause significant interference affecting the **DuoBravo**. As a precautionary measure please make sure that the power cord of the laptop is as far away as possible from the lead wires of the patient. If interference persists, remove the power cable from your computer and use it with its battery.
- If device operation occurs close to short wave, microwave, cellular or other wireless equipment, instability of the values of the stimulator could occur causing a temporary reduction in current.
- If the device is connected to a surgical high frequency apparatus at the same time, this could result in burn marks underneath the electrodes.
- The application of the electrodes near the thorax can increase the risk of cardiac fibrillation.
- In the case of pregnancy or at persons with implants, electrical stimulation should only be applied after consultation with an experienced doctor.
- Stimulation in the area of tumors is allowed only by instruction of an experienced physician.
- If in the case of treatment with the incontinence probes, pain or irritation appear in the area of the pelvis, or bleeding in-between menstruation, interrupt the therapy and consult your doctor.
- If high frequency is combined with a great pulsewidth, please stimulate with small intensity only.
- If you have swallowed any part of the device or any of its accessories, please contact a doctor immediately.
- TENS and muscle stimulation should be used during menstruation or by pregnant women as well as by children under the guidance of a doctor or therapist only.
- Please don't use a probe during menstrual period or pregnancy.
- The **DuoBravo** may not be operated if the battery compartment is not closed. Before opening the battery compartment, make sure that there are no cables in the jacks at the head of the unit.

Please, keep this device stored out of the reach of children. Do not leave the lead wires in reach of children, since strangulation is possible due to the length of the lead wires.

Advice on Application of Electrodes

Make sure that **DuoBravo** is turned off (no indication visible on the LCD display).

Now connect the single poled ends of the lead wires with the electrodes.

The output plugs of the electrode cables will be plugged into the channels A or B of the

DuoBravo. The reference cable will be connected to its jack as well if EMG will be used.

Furthermore make sure, that all plugs fit completely into the electrode sockets as well as the device sockets.

If you are using a probe, lubricate it with water or a gel suitable for electrotherapy.

Please don't use any oil, ointment or cream, since all these have an isolating effect.

Now place the electrodes or the probe according to the instructions of your doctor or

physiotherapist. The self-adhesive skin electrodes are provided with a sticky gel and can be

re-used after removal from the skin. Replace these electrodes when the adhesive force is weak.

Electrodes which are damaged due to careless handling may not be used again.

Electrodes with damaged cable insulation (e.g. visible copper wire) may not be used.

Electrodes with damaged, missing or stained aluminium coating or carbon layer, also with insufficient adhesive gel may not be used.

Electrodes may not be altered or tampered with (e.g. by scissors or other tools).

Do not pull the electrodes by force on the electrode cable. To remove the electrodes from the skin, please lift the electrode at the edge towards the middle.

Do not use electrodes with ointments or creams (especially which enhance the blood flow or act as local anaesthesia).

Apply the electrodes on clean and unwounded skin only.

In the case of strong bodily hair growth, the hair should be shaved or shortened with scissors.

Neither stretch the skin nor the electrodes when applying the electrodes.

Effective current densities of more than 2 mA/cm² require an increased attention of the user with regard to the intensity adjustment.

For standard treatment we recommend the use of our self-adhesive electrodes with a minimum area of 40x40 mm. This size is suited for the reference electrode as well. For best results please make sure that it will be placed on a bone near the treatment area, not on muscles.

For treatment of the muscles of the pelvic floor we recommend the use of our vaginal- (IncoProbe V) or rectal-probes (IncoProbe R).

After using a probe, please clean it carefully with a mild detergent without alcohol and dry it completely after each use.

After all electrodes have been removed, wash the skin with mild soap and rinse thoroughly. Then dry your skin.

All electrodes or probes are made for use of one single person only.

Skin electrodes as well as probes can contain amounts of nickel. Allergy sufferers should check this before using. Should skin irritation occur whilst using the **DuoBravo**, please interrupt the application and inform your doctor, so that an allergic reaction can be excluded.

Maintenance

DuoBravo has been designed to be maintenance free. Nevertheless, please adhere to the following advices:

If needed, please clean **DuoBravo** with a mild detergent, water and a damp cloth. Do not expose **DuoBravo** to extreme humidity and do not submerge under water.

You may clean the cables with a damp cloth. You may also from time to time treat the cables with talcum powder in order to prevent cracks and to further on going usage.

A clean cloth, moistened with an alcohol free disinfection liquid can be used to clean the device. Please follow all instructions of the manufacturer of the liquid.

After termination of the therapy, **DuoBravo** can be re-used by other patients. Please clean the device with a mild disinfectant cleaner.

The **DuoBravo** does not require calibration. The **DuoBravo** is not repairable and contains no user serveable parts. No modification to the **DuoBravo** is allowed.

Warranty

MTR+ Vertriebs GmbH provides a 24 month warranty, commencing from the date of our delivery (invoice date), that in case of failure, the **DuoBravo** will be serviced or replaced free of charge. All returns must first be authorised by MTR+ Vertriebs GmbH in advance.

Excluded from this warranty are failures of the **DuoBravo** which result from excessive use, careless or improper handling as well as damage by force.

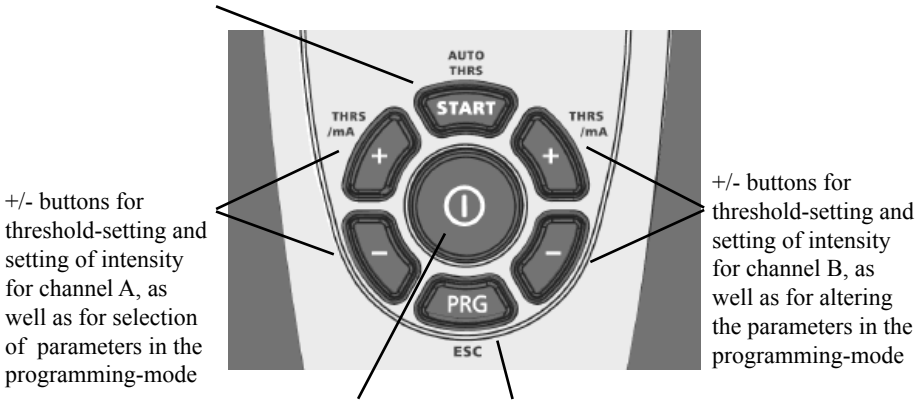
The warranty loses its lawful meaning if the producer MTR+ Vertriebs GmbH has not given its consent in writing prior to repairworks or servicing being carried out. Without this written consent the manufacturer will not return the device.

The manufacturer reserves the right of technical changes, which may serve to increase treatment success, the functionality of the device or its accessories.

Operating Elements


The **START**-button allows the choice of automatic (press) or manual (hold the button for 3 seconds) threshold-setting if an EMG-or ETS-program is chosen. The program will start after the **START**-button is released.

Central On-/Off-button

PRG-button for choice of program and for storing the individual parameters. Hold PRG for one second to select next phase.

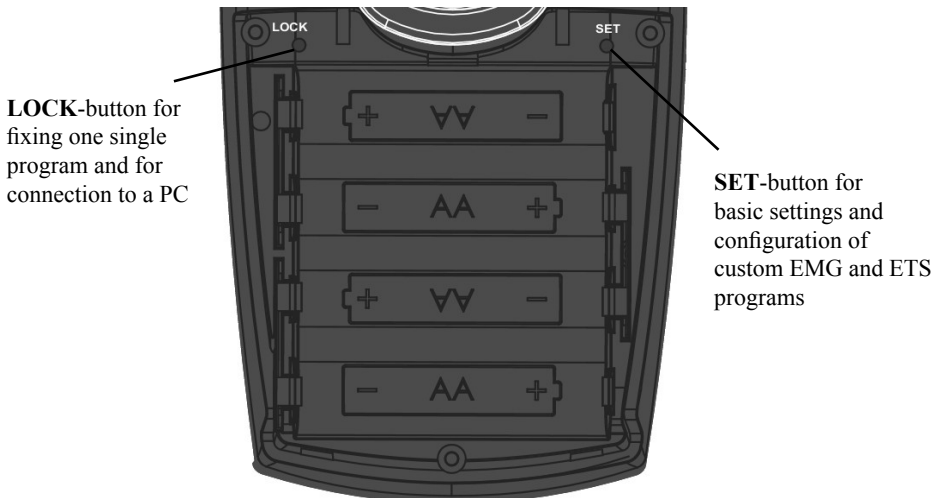
Battery Compartment

Scheme for placement of 4 batteries type AA

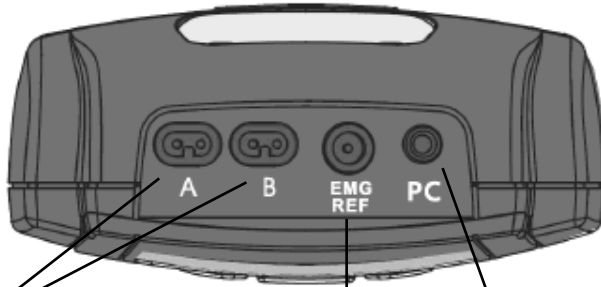Connections

Jacks to connect the electrode-cables of channel A and/or B, each suited for surface-electrodes or probes.

Jack for the fiber optic cable to connect via FOC-adapter to PC

Jack for the single EMG-reference cable (skin-electrode)

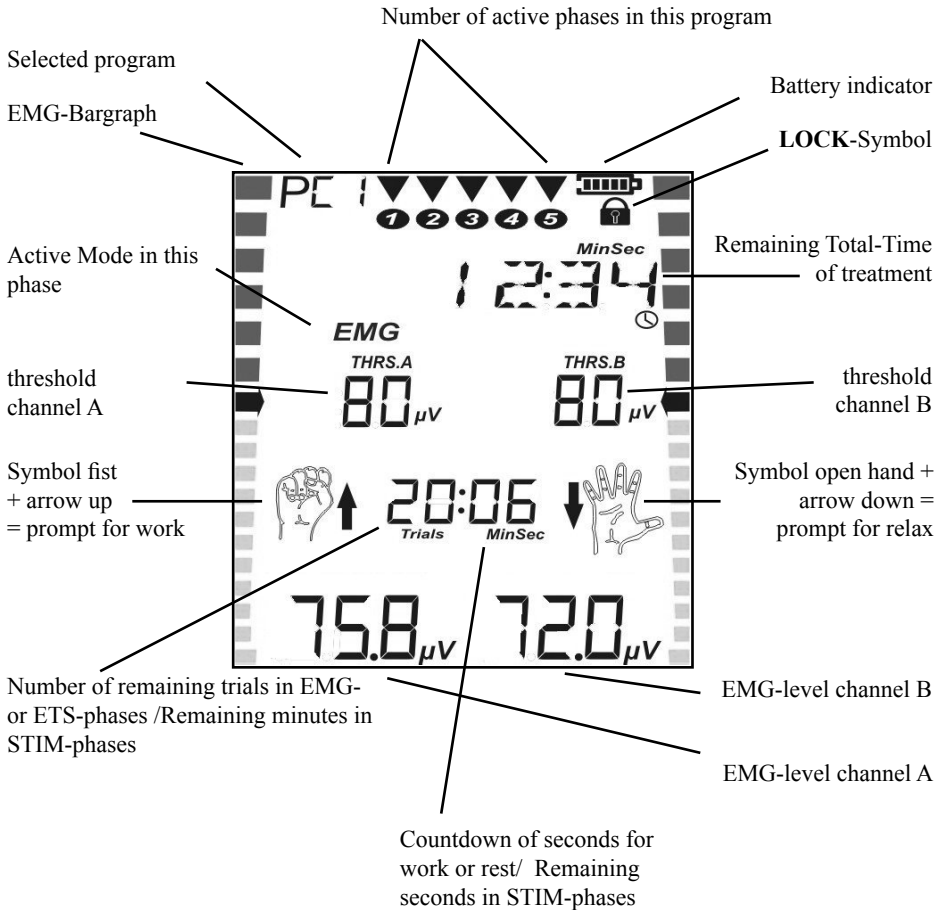
Attention: Even if the device is used with a probe, please always connect the single reference-cable.

The Display

After selecting a program with the **PRG**-button **DuoBravo** shows information about those parameters in its "Home Screen" that are used in the first phase of the program (stimulation frequency, pulse width, number of trials, threshold, etc.). Start the treatment by pressing or holding **START**, the display changes to work-screen and shows necessary information for treatment in the active phase.

Work-Screen (in EMG-mode)

The Info-Line

The info line in the upper part of the screen shows always the remaining time of the entire program during treatment.

Outside a treatment session, it provides more additional information:

- When selecting a program, the target of the treatment is indicated in abbreviations. (REHAB = Rehabilitation / MIXED = mixed incontinence, etc.).
- If the circuit is interrupted, caused by removal of an electrode or a broken cable, the term CABLE flashes in the info line.
- It shows custom values to be set in plain text.

The info line displays all custom settings for configuring the device. In the following you find all expressions and abbreviations that are used in the programming-loop in alphabetical order:

CH.B. ON / CH.B. OFF: Channel B is switched on or off

DELAY: Possibly delay of current between channel A and B if synchronous stimulation is selected

EMG OFF / EMG ON: EMG-measuring during rest-period of STIM-phases

FB. CONT: Continuous acoustic feedback

FB. OFF: No acoustic feedback

FB.ABOVE / FB.BELOW: Feedback above or below the threshold

FREQ.: Number of stimulation pulses per second

INHIB.OF: Inhibition switched off

INHIB.ON: Inhibition of feedback on channel B

NARROW: Narrow bandpass filter used

PC FOC: Connection to PC by fiber optic cable with USB dongle

PC OFF: Connection to PC is switched off

PC SWC: Connection to PC by secured wireless connection

PULSE: Duration of each single pulse in μ sec

RAMP DN.: Time needed from the end of complete intensity to complete end of current

RAMP UP: Time to reach complete intensity of stimulation

REST: Duration of rest-sequences in seconds

STIM.TME.: Duration of each single stimulation in seconds

THRESH: Threshold-adjustment during treatment automatic (AUTO) or manually (MAN)

TIME: Duration of a STIM-phase in minutes

TRIALS: Number of repetitions

TRIGGER: Time that the threshold must be reached to trigger stimulation

VOLUME: Sound volume of beeps and spoken instructions

WIDE: Wide bandpass filter used

WORK: Duration of work-sequences in seconds

Basic Settings

In all programs, even in the preset programs, individual settings can be made as follows. After you have put in four batteries type AA (s.p. 63) and you have turned on the device with the **ON/OFF**-button, you enter the configuration mode by pressing the **SET** button in the battery compartment for three seconds. Now you can adjust the settings for the first phase of the selected program. To configure the next phase, **SET** should be pressed again.

The parameter that can be changed is shown in the info line and can be altered by pressing the + button of channel B. Pressing the + button of channel A will confirm the choice and the next variable parameter is displayed. This system - changing settings with B / storage and stepping forward with A - remains always the same during the entire process.

Please note that the following settings you make here, are automatically stored for all subsequent phases and all other programs as well. Of course you can change any setting at any time in the same way again.

STIM-phases

- Channel B can be switched on or off.
- In phases with work/rest-rhythm it can be decided
 - a) whether EMG shall be measured during the rest-period and
 - b) which filter shall be used
- PC-connection can be chosen or switched off.

EMG-phases or ETS-phases

- Sound-volume
- Automatic threshold-adjustment on or off
- Wide- or narrow-filter for channel A
- Channel B switched on or off
- Filter for channel B if it is in use
- PC-connection can be chosen or switched off

In the customizable programs PC1 - PC4 of course you have the chance to determine numerous parameters more in the same way as described before (s.p. 75-79).

Setup Real Time Clock

Before first use of **DuoBravo** the “Real Time Clock” that is integrated should be set. It’s necessary for later therapy-control. **DuoBravo** has a small assistance battery that will support the clock with energy for several years. For this reason, there is no need to set again, even if the 4 main batteries have been substituted. But we recommend to check and readjust if necessary.

For setting time and date, please select any program and press the **SET**-button for at least 3 seconds. You’ll reach the configuration-mode. Now press **SET** once again for 10 seconds. On screen HOUR is indicated and you can set the hour with the +button of channel B (00-23).

By pressing the +button of channel A you confirm this choice and step forward to the setting of MINUTE. This is indicated on the screen as well, giving the opportunity to select the minutes via +button of channel B in the same way as before.

This way you are able to set one after the other:

- HOUR
- MINUTE
- DAY
- MONTH and
- YEAR

If all settings are finished, please press the **ESC**-button on the frontside of the device. All values will be automatically stored and the clock starts working.

Setup Language

DuoBravo is able to show 6 different languages. If you are in the setting-mode of the RTC (see above), please press the **SET**-button for 10 new seconds once again. Now you can select German-, English-, French-, Spanish-, Italian- or Polish-language with the B-buttons. Finally you store your choice with the **ESC**-button.

Preparation of Therapy

1. Open the battery compartment at the back by pushing down the lid and insert 4 batteries type AA by hand according to the instructions on page 63. Then shut the battery-compartment.
2. Please make sure, that **DuoBravo** is switched off (no indication in LCD-display). Connect the single poled ends of the electrodes plugs with the surface-electrodes or with a probe.
3. Insert one electrode-cable into the jack named A in the head of the device. If you want to use both channels, insert the second cable into jack B.
4. Additionally, please put the black round EMG-reference plug into the bigger round jack, named "EMG-REF" and connect the other ends to a skin-electrode.
5. Furthermore, make sure that all plugs fit completely into the electrode sockets as well as the device sockets.
6. If you are using a probe, lubricate it with water or a gel suitable for electrotherapy. Don't use oil, ointment or cream, since all these have an insulating effect.
7. Now place all electrodes according to the instructions of your therapist on clean undamaged skin only. If you are using a probe, place the single reference-electrode onto the hip bone and then insert the probe.
8. Now switch on **DuoBravo** by pressing the central **ON/OFF**-button once. You can see the version of the device for 3 seconds before the last used program appears, showing its parameters.
9. If the device is not locked to one single program, you can select your kind of treatment by repeatedly pressing the **PRG**-button. A great variety of programs with one up to five phases each are available. Within one single program the phases can be programmed in different modes.
10. After selecting a program, you always see the "Home Screen" first, with all information regarding the parameters, that are contained in the first phase of this program. If the chosen program has an EMG- or ETS-phase, the manual threshold-setting is possible. In this case, the treatment starts by holding the **START**-button for more than 1 second. If you prefer the automatic threshold-setting, no adjustments have to be done and treatment starts by short pressure of the **START**-button. Follow the work/rest prompts, so the device can automatically select the threshold setting for you.
11. EMG is susceptible for interference. For best results, do not use lead wires longer than 1.5 meters. Tie excess cables of the lead wires together and keep close to, preferably touching the body.

The Different Phases

DuoBravo offers programs that can consist of one or more phases. There are basically three different types of these phases which can be therapeutically combined in a useful way.

- a) **EMG** (= Electromyography) - purely active biofeedback training is carried out. The patient tightens the muscle according to the requirements in the display for the specified time and then releases it again. The device gives an acoustic and visual feedback about the strength and duration of contraction and relaxation. If the relaxation is target of the treatment, the positive signal will be given in the active period (that is indicated by the fist) as well, if muscle tension will be below the threshold. In an EMG-phase the device only measures and doesn't give any current.
- b) **STIM** - In this phase the muscle is stimulated passively. Here is a distinction between:
- Continuous current (CON), which does not include breaks and always remains the same.
 - Modulated current, which also flows without a break, but frequency and pulse width change in a predetermined rhythm. This way, the feeling of rising and falling intensity is generated.
 - Current in work/rest-rhythm, where a muscle is stimulated in each work phase to contract as strong as possible. In all subsequent rest phases the muscle shall be relaxed as far as possible, to avoid strain.

The effect of current delivered by the device will vary depending on configuration. It may have circulatory-enhancing, detoxifying, relaxing, strengthening and pain-reducing effect. During the stimulation **DuoBravo** cannot measure EMG, but the muscle tone can be measured in the relaxation sequences of work/rest-periods. This control may be necessary to avoid any overstrain of the treated muscle.

- c) **ETS** (= EMG-triggered stimulation) - this is a combination of active biofeedback training (see above - EMG) and immediately afterwards passive electrical stimulation (see above - STIM). If the patient succeeds to reach a preset threshold for the muscle tension by his own activities, the device automatically transmits current that supports the muscle to contract. The movement generated by the current, is initiated by the brain this way. It's the target to improve the force of the own muscle contraction. In neurological treatment, it is often intended to remove interferences in neural pathways, so that the muscle can be actively moved later - without the help of a device.

Therapy with DuoBravo

DuoBravo is a versatile device, that measures, compares, displays results, adapts itself, gives current, calculates statistics ... and much more. Nevertheless it is the fundamental target, to be simple in use for the patient.

For this reason, each kind of therapy is possible without effort in preparation. Beside 4 free customizable programs in each version, the following number of pre-set programs are available, that can be used for therapy at once, without any further adjustment.

- **DuoBravo** U (Urology) = 12 programs
- **DuoBravo** N (Neurology) = 12 programs
- **DuoBravo** U + N (Urology - Neurology) = 22 programs

Use of pre-set programs

1. Connect the electrodes to the device (s.p. 64)
2. Place electrodes or probe according to the instructions of your therapist
3. Switch on **DuoBravo** with the central **ON/OFF**-button
4. Select the desired program with the **PRG**-button
5. Press **START**
 - In phases with pure passive stimulation you'll hear a short melody and mA are flashing, showing that it can be adjusted now with the +button of any used channel. The intensity will increase 1 mA with each press - please adjust current as far as it is possible without any pain. 5 seconds after last adjustment stimulation starts. Please note that this increase is only possible, if the +button is active (s.p. 56 - Safety Adjustments).
 - In EMG- or ETS-phases threshold will be always find out automatically by doing two short work/rest-periods. If that's not intended you can switch off this function by holding the **START**-button for three seconds.
 - An EMG-phase would start with the first rest-period now. If you are using an ETS-phase the short melody sounds and flashing mA prompt for intensity-adjustment.

If the selected program has several phases, each phase-end will be indicated by signal beeps. Please note that stimulation-intensity will fall down to 8 mA each time a new phase starts. The intensity will need to be adjusted accordingly.

Therapy with DuoBravo

Therapy in a phase with pure stimulation

In phases with work/rest-rhythm you can support treatment success by active muscle contraction during work-period effectively. Please take care that the muscle will relax in rest-periods completely. In display **DuoBravo** shows the fist as symbol for “work” and the open hand for “rest”.

At the beginning of each stimulation phase you hear a short melody and the mA-indicator is flashing in the display, showing that intensity should be adjusted now with the +/- buttons of each active channel.

The stimulation intensity is indicated with numbers in the bottom-line of the display and is shown by the lateral bar graphs on the screen as well. The more bars are active, the higher the current intensity.

If you want to interrupt the treatment of a stimulation-phase, please press **START** and the current will stop immediately. In the infoline of the display the term PAUSE appears. You can continue the treatment by pressing **START** once more. Then stimulation starts again for the remaining treatment-time with the same intensity it had before it paused.

In the rest-periods of programs with work/rest-rhythm, EMG can be measured to see if the treated muscle is relaxed enough to avoid overloading (s.p. 67 and 76). The corresponding values are shown in the bottom line of the display.

Therapy with DuoBravo

Therapy in an EMG-phase

The device indicates a pre-set rhythm of work and rest, that the patient shall execute as precisely as possible.

During work-period the symbol of the fist with up arrow is seen on the screen. The feedback regarding the success of your own muscular activities is given by the increasing numbers in the lowest line and by the yellow/green bargraph on each side of the display. Reaching the green area shows that the threshold - regardless whether it has been set automatically or has been set manually - is met and the trial was successful. Additionally this success is indicated by signal beeps. Please take care to keep your efforts alive, until the time, that is indicated beside the fist, has gone to zero and the symbol of the open hand with arrow down appears. Then relax the muscle as far as possible.

If you have chosen automatic threshold adjustment during treatment, threshold would be reduced if you are not successful and it would be easier to succeed in the next trial. If this next trial is successful again, threshold would newly increase.

The process of work and subsequent relaxation shall be repeated again and again - until the number of remaining trials is indicated 00 (s. p. 65). Please note that the display doesn't change before the relaxation-phase is completed.

If relaxation is target of the treatment, the positive signal will be given in the active period (fist) as well, if muscle tension will be below the threshold.

Therapy with DuoBravo

Therapy in an ETS-phase

During an ETS-phase you behave identically to an EMG-phase, which means, that you follow the rhythm of work and rest, given by the device, with your muscular activities. If you succeed reaching the threshold the term BRAVO is indicated in the infoline of the display and the device stimulates your muscles automatically. In this case your muscle will be contracted passively. Please support the treatment success by keeping your effort alive during stimulation until there is no current-flow anymore. Then the rest-period starts and you should relax the muscle as far as possible.

If you don't reach the threshold in one of the trials, you will not receive stimulation and the rest-period starts at once after termination of the work-period.

If you have chosen automatic threshold adjustment during treatment (s.p. 81), threshold will be reduced in this case and it would be easier to succeed in the next trial. If this next trial is successful again, the threshold will increase.

The whole process of work/current and subsequent relaxation shall be repeated again and again - until the number of remaining trials is indicated 00 (s. p. 65). Please note that the display doesn't change before the relaxation phase is completed.

Therapy in combined programs

Programs, which contain several modes (STIM and/or EMG and/or ETS), will always start with automatic threshold-setting, if you don't skip it (s.p. 80). When threshold is set you can adjust the current-intensity with the +/--buttons of the used channel(s) if the program contains any STIM-phase. After this, each step is identical to the description above.

Additional Options for Configuration of DuoBravo

Additionally to the handling of pre-set programs as described in the previous chapter, **DuoBravo** offers a great variety of possibilities to adapt the therapy-parameters according to individual needs.

This concerns the pre-set programs that can vary in regard of the basic adjustments as well as the 4 free customizable programs, in which the experienced therapist or user is able to create completely new programs with up to 5 different phases.

Configuration of Free Programs

Creating a new program is not difficult, because the programming always follows the same easy scheme. The device leads in a loop to each parameter that can be chosen - indicated by flashing bars.

1. Switch on **DuoBravo** with the **ON/OFF**-button
2. Select PC1, PC2, PC3 or PC4 with the **PRG**-button. In **DuoBravo U** and **DuoBravo N** these custom programs are 13-16, while in **DuoBravo U+N** the programs 23-26 are customizable. In the left upper corner of the screen PC1-PC4 will be indicated.
3. Keep the **SET**-button in the battery-compartment pressed for 3-5 seconds. Then **DuoBravo** changes to programming mode and the bars beside the first parameter that can be set are flashing.
4. In the following the right “+/-”buttons (channel B) are responsible for changing the flashing parameter, while the left “+” button (channel A) is used to store the choice and step forward to the next parameter. With the left “-”button it is possible to go one step back.

With the described scheme “right side changing / left side storing and step forward” all necessary decisions can be met. If the first parameter is shown a second time, the configuration of this phase is complete and by short pressing of **SET** the next phase is available. If you store “OFF” then, this and all following phases are deleted. If all phases are ready programmed, the whole program can be stored with **ESC**. This configuration will stay available until a new programming is done on this place or the whole device is reset to its original state.

Parameters of Free Programs

Programming PC1

PC1 is a multi-phase program for passive stimulation only. With A+ the following parameters can be chosen and then altered by B+, if necessary.

- **Type of stimulation** - (indicated in display: CON / SYN / ALT) - sets whether stimulation shall be continuously or in work/rest-rhythm.

If **CON** (= continuous current) is selected, you can afterwards only decide about:

- **Duration of phase** (PH.TIME)
- **Frequency** (FREQU.) - fixes the number of pulses or pulse-groups per second. Frequency is indicated with Hz.
- **Pulse width** (PULSE) - names the duration of each single pulse and is measured in microseconds (μ S).
- **Channel B** (CH.B. ON / CH.B. OFF) switched on or off
- and **PC-Interface** (PC SWC / PC FOC / PC OFF) - here you can decide whether all treatment data will directly be transferred during treatment to a PC with optional therapist-software via fiber optic cable, alternatively by secured wireless connection or not. During home-therapy PC OFF should always be selected to save battery-power.

If you want to use both channels in a work/rest-rhythm simultaneously, please select **SYN**. In this case the following parameters can be set:

- **Duration of phase** (indicated in display s.a.)
- **Frequency** (s.a.)
- **Pulse width** (s.a.)
- **Duration of work-period** (WORK) - determines the time that one single muscle contraction should be hold
- **Duration of rest-period** (REST) - determines the time of relaxing after contraction
- **Ramp up-time** (RAMP UP) - means the time (in secs.) of increasing current
- **Ramp down-time** (RAMP DN) - means the time (in secs.) of decreasing current
- **Channel B** (CH.B. ON / CH.B. OFF) switched on or off
- **Delay of channel B** (DELAY) - only if needed
- **Measuring of EMG in rest-periods** (EMG OFF / WIDE / NARROW)
- and the **PC-Interface** (s.a.)

If in work/rest-rhythm both channels should not work simultaneously, but one after the other, please select **ALT**. Then you have to set all parameters identically to **SYN** - with exception of delay.

If programming of one phase is ready please press **SET** to go to the next phase.

The range of settings and the size of possible steps is shown on page 79.

Parameters of Free Programs

Programming PC2

PC2 is a single-phase EMG-program. With A+ the following parameters can be chosen and then altered by B+, if necessary.

- **Volume** - (indicated in display: VOLUME) - sets the volume of sounds for feedback in steps 1-10
- **Duration of work-period** (WORK) - determines the time one single muscle contraction should be hold
- **Duration of rest-period** (REST) - determines the time of relaxing after contraction
- **Number of trials** (TRIALS) - names the number of active exercises that shall be done
- **Threshold** (THRESH) - defines whether automatic threshold-adjustment during therapy is switched on (AUTO) or off (MAN).
- **EMG Sound Feedback** – a) Off (FB.OFF). On (FB.ON), Above Threshold (FB.ABOVE) – normally used when training muscle force, Below Threshold (FB.ABOVE) – normally used for relaxation training.
- **Wide or narrow filter** (WIDE / NARROW) for channel A - this choice is necessary to avoid wrong measurement, caused by heart beat (60-120 Hz). Narrow filter should be used for the torso, wide filter should be used for legs and arms.
- **Channel B** (CH.B. ON / CH.B. OFF) switched on or off
- **Wide or narrow filter** for channel B
- **Feedback inhibition** (INHIB.OF / INHIB.ON) - will be selected, if the user shall get positive feedback only, if he will be above threshold of channel A **and** below threshold of channel B. This way it is tried to avoid wrong EMG-results, caused by wrong muscles, that are contracted near the measured muscles.
- and **PC-Interface** (PC SWC / PC FOC / PC OFF) - here you can decide whether all treatment data will directly be transfered during treatment to a PC with optional therapist-software via fiber optic cable, alternatively by secured wireless connection or not. During home-therapy PC OFF should always be selected to save battery-power.

Parameters of Free Programs

Programming PC3

PC3 is a single-phase ETS-program. With A+ the following parameters can be chosen and then altered by B+, if necessary.

- **Volume** - (indicated in display: VOLUME) - sets the volume of sounds for feedback in steps 1-10.
- **Duration of work-period** (WORK) - determines the time that one single muscle contraction should be hold.
- **Duration of rest-period** (REST) - determines the time of relaxing after contraction.
- **Number of trials** (TRIALS) - names the number of active exercises that shall be done.
- **Duration of stimulation-period** (STIM.TME) - names the time of current-flow if threshold was reached.
- **Frequency** (FREQU.) - fixes the number of pulses or pulse-groups per second. Frequency is indicated with Hz.
- **Pulse width** (PULSE) - names the duration of each single pulse and is measured in microseconds (μ S).
- **Ramp up-time** (RAMP UP) - means the time of increasing current.
- **Ramp down-time** (RAMP DN) - means the time of decreasing current.
- **Threshold** (THRESH) - defines whether automatic threshold-adjustment during therapy is switched on (AUTO) or off (MAN).
- **EMG Sound Feedback** – a) Off (FB.OFF). On (FB.ON), Above Threshold (FB.ABOVE) – normally used when training muscle force, Below Threshold (FB.ABOVE) – normally used for relaxation training.
- **Wide or narrow filter** (WIDE / NARROW) for channel A - this choice is necessary to avoid wrong measurement, caused by heart beat. If the patient measures EMG near the heart you should use NARROW-filter.
- **Channel B** (CH.B. ON / CH.B. OFF) switched on or off
- **Wide or narrow filter** for channel B
- **Trigger-time** (TRIGGER) - is the time that muscle-tension of a user must remain about the threshold in order to trigger the current. There are two settings available. 0.25 seconds is suitable for urology and avoid excess muscle twitches triggering the current. The patient must hold the EMG value above the threshold for 0.25 seconds or more to trigger the current. 0.01 seconds is suitable for neurological treatments and can detect the smallest muscle twitch which is above the threshold. The patient only needs to hold the EMG value above the threshold for 0.01 seconds to trigger the current.
- and **PC-Interface** (PC SWC / PC FOC / PC OFF) - here you can decide whether all treatment data will directly be transferred during treatment to a PC with optional therapist-software via fiber optic cable, alternatively by secured wireless connection or not. During home-therapy PC OFF should always be selected to save battery-power.

Parameters of Free Programs

Programming PC4

PC4 is a multi-phase program, in which each of the max. 5 phases can be configured in STIM-, EMG- or ETS-mode. Press SET to select consecutive phases.

If you want to change a configured program and you do not need all the phases, please set the first phase that is no longer needed to “OFF”. This and all further phases are deleted.

Please note that for safety reasons each phase containing stimulation will start with 8 mA max. Intensity has to be adjusted newly to a therapeutic effective level without causing any pain.

The settings can be made as follows.

- Frequency: 1, 2, 3, 4, 5, 6, 7, 8, 9, 10, 11, 12, 13, 14, 15, 16, 17, 18, 19, 20 Hz, then increasing in 5 Hz steps until 50 Hz and in steps of 10 Hz up to max. 100 Hz.
- Pulse width: 50 to 450 μ S in steps of 10 microseconds
- Work time: 0,5-99 seconds in steps of 1 second
- Ascent time: 0,1-9,9 seconds in steps of 1 second (Ramp up)
- Descent time: 0,1-9,9 seconds in steps of 1 second (Ramp down)
- Rest time: 0,5-99 seconds in steps of 1 second
- Delay: 0,0-4,0 seconds in steps of 0,1 second
- Session time: 1-99 minutes in steps of 1 minute

Lock-Button

If the therapist wants to permit for his patient the use of one single program only, all other programs can be locked out.

To do this, please select the desired program and press the **LOCK**-button with a thin pin. For this you can use the electrode connector. You'll hear a short beep and the symbol of a padlock appears on screen.

If you want to use any other program later, or if you need to unlock all programs, **LOCK** must be pressed in Home Screen once again and **CONN-PC** (= PC-connection) or **NO.STATS** (= no statistics) will appear. Keep the **PRG** button pressed for 3 seconds, the display changes to Home Screen and all programs are available. Press **PRG** for less than one second to return to the Home Screen with the unit still locked.

Threshold

The threshold is a target value that shall be achieved by personal effort. It is measured in microvolts and displayed for both channels in the middle line of the screen. The arrows in the middle of the bargraph on the left and right side of the display represent the thresholds. Since all programs with EMG or ETS phases refer to a threshold, it is necessary to set the threshold for all programs with EMG or ETS phases.

Setting of threshold before start of treatment

By default, the threshold is determined automatically when **START** is pressed if any phase in a program is EMG or ETS.

Pressing the **START** button causes the appearance of the open hand in the display, while the time-indicator for this rest-period runs to 0. At 0, the display switches to the symbol of the fist and the time for this 5-second period of work runs to zero again. **In this short work period, you should try to contract the muscle with the maximum possible effort.** Subsequently a rest period follows and a further work-period. During these two initial work/rest trials for threshold-detection the word **TEST** is indicated in the infoline. The threshold is calculated automatically after the second phase of work, and is set at 80% of the average value of the period with higher performance. Please note that the first second of each work-phase is not relevant for the average value.

Now the treatment begins with the threshold set and the total time counts down in the infoline.

Alternatively, the automatic setting can be skipped. This is achieved by holding the **START** button pressed for more than one second. In this case, the threshold of the last treatment appears and could be altered manually by pressing the +/- buttons before starting the program.

Threshold

Automatic adjustment of threshold during treatment

Once the threshold is set, it will stay the same for the entire duration of the treatment until you make a new setting - except if you have selected “automatic threshold adjustment.” In this case, the threshold would be adjusted during the treatment according to your performance.


EMG-Phase

In an EMG-Phase the new threshold will be set to 80% of the average work that have been done in the last trial before.

ETS-Phase

If you reach the threshold during a trial, it increases for the next attempt and if you don't succeed to reach the threshold, it goes down slightly. The stepsize of increase or decrease depends on the time that was needed to reach the threshold in the last trial before.

The following graph will illustrate the principle of adaptation in an ETS-phase.

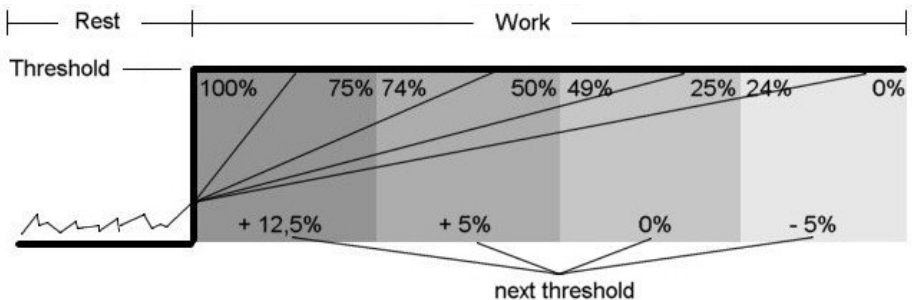If the threshold is reached in the first quarter of the time, it increases by 12.5%.

If the threshold is reached in the second quarter of the time, it increases by 5%.

If the threshold is reached only in the third quarter of the time, it does not increase or decrease.

If the threshold is reached only in the last quarter of the time, it decreases by 5%.

If the threshold is not reached, it decreases by 12.5%.

Therapy-Control

DuoBravo displays statistics after each program has been completed. All values of this completed session, which are named in pages 83-84 will be indicated on the screen automatically. With A+ you can scroll through the statistics. **DuoBravo** saves these statistics to its memory for download to the optional software.

The treatment data is stored for up to 365 days. The additional software is required to transfer the stored data to a PC. An electronic patient record, which allows the graphical presentation of the treatment progress, can easily be created. This optional software is very helpful for therapists for statistical and/or scientific reports. To transfer the data please insert the USB dongle, which is supplied with the software, into the computer. If the USB dongle supports a fiber optic cable, attach the fiber optic cable between the **DuoBravo** and the USB Dongle. The fiber optic cable is connected to the "PC" plug in the head of the unit (see page 64). The USB Dongle should be plugged into your computer and connected to the fiber optic cable (if required) before starting the transfer.

Then the transfer of stored data can be done as follows:

- Start the optional software on your PC.
- Create or select the patient.
- Switch on the **DuoBravo**.
- Press the LOCK-button once, if the device was locked – or twice if it was unlocked.
- You will see "CONN-PC" in the display - or "NO.STATS" if the memory is empty.
- The data transfer starts automatically and your PC shows a message that it is receiving the statistics. The transfer may take several minutes depending on the amount of data stored in the **DuoBravo** memory.
- When the statistics have been uploaded, the software will confirm that the transfer is complete.

The values in the therapeutic memory are retained after they are read, so it is possible to check therapy at any time, without losing information.

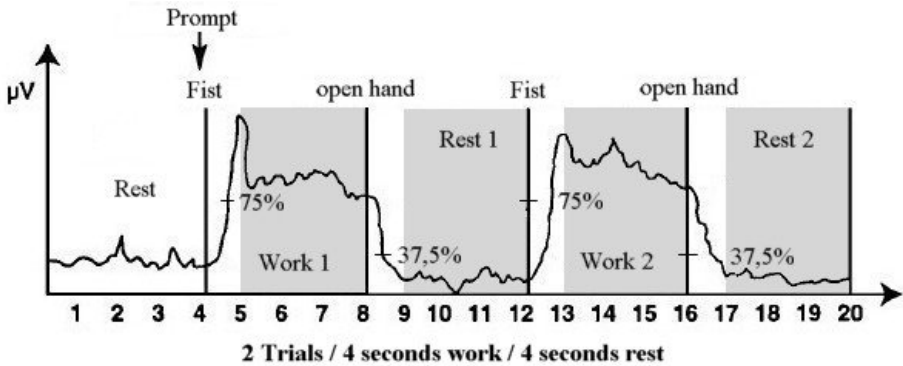
If the memory needs to be deleted completely, it can be done with the following button-combination. Press the LOCK button in the battery compartment until CONN-PC is displayed and then successively SET, B +, B- and SET once again. You will hear a sound to confirm the statistics are being deleted. This action will also reset all custom settings in the freely customizable programs back to the original state.

The deletion of the memory is required:

- before the device is used by a new patient
- 1 year after initiation of the treatment at the latest

Statistics

After termination of a treatment session **DuoBravo** shows statistics on its screen as follows.

WORK.AVG - Work Average:

The average value in microvolts of all the work-segments excluding the first second of each segment.

REST.AVG - Rest Average:

The average value of the rest-segments excluding the first second.

ONSET - Average Onset:

The average time taken after each respective “**work**” prompt to reach 75% of the average value of the previous work period. If the onset of trial(s) is longer than 2 seconds, it will be rejected. The display will indicate the average of only those trials, where onset lasts 2 seconds or less. The number of valid onsets is displayed above the onset value. If the number of valid onsets is lower than the number of trials, it means some onsets were rejected.

RELEASE - Average Release:

The average time taken, after the “**rest**” prompt to reach 37.5% of the average EMG from the previous work period. If any release of the trials is longer than 2 seconds then it will be rejected. The display will indicate the average of only those trials where release lasts 2 seconds or less. The number of valid releases is displayed above the release value. If the number of valid releases is lower than number of trials, it means some releases were rejected. If all the releases are above 2 seconds, ABOVE2 will appear on the screen.

W/R PEAK - Work/Rest Peak:

It is the peak value during the whole cycle of all the trials.

Statistics

WORK DEV - Average deviation (per second) of work:

It is the average deviation of the quarter-second samples from the average value in the second in which they occur (excluding the first second of each work-segment). If deviation is more than 20% it is considered problematic.

REST DEV - Average deviation (per second) of rest.

The average deviation of the quarter-second samples from the average value in the second in which they occur (excluding the first second of each rest-segment). If deviation is more than 20% it is considered problematic.

INT.STIM - Average intensity of current:

Average intensity of current in all phases with pure stimulation.

INT.ETS - Average intensity of current:

Average intensity of current in all phases with ETS.

SCORE - Percentage of success:

The closer this value reaches 100%, the faster the patient has average achieved the threshold during the ETS-treatment and therefore the better is his muscle condition.

Programs Urology

Progr.: 01		EMG				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Mode		EMG				
Number of cycles		40				
Work time	secs	8				
Rest time	secs	8				
Progr.: 02		ETS				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Mode		ETS				
Number of cycles		25				
Frequency work	Hz	35				
Pulse duration	µS	300				
Ramp up time	secs	1,5				
Ramp down time	secs	1,5				
Work time	secs	8				
Stim time	secs	5				
Rest time	secs	12				
Progr.: 03		Stress 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5				
Number of cycles			25			
Mode		W/R	EMG			
Frequency work	Hz	35				
Pulse duration	µS	300				
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	5	5			
Rest time	secs	10	10			

Programs Urology

Progr.: 04		Stress 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5				
Number of cycles			30			
Mode		W/R	ETS			
Frequency work	Hz	35	35			
Pulse duration	μS	300	300			
Ramp up time	secs	1	1			
Ramp down time	secs	1	1			
Work time	secs	5	5			
Stim time	secs		5			
Rest time	secs	15	15			
Progr.: 05		Stress 3				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5		5		
Number of cycles			20		20	
Mode		W/R	EMG	W/R	EMG	
Frequency work	Hz	35		35		
Pulse duration	μS	250		250		
Ramp up time	secs	1		1		
Ramp down time	secs	1		1		
Work time	secs	5	5	5	5	
Rest time	secs	10	10	10	10	
Progr.: 06		Stress 4				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5				
Number of cycles			20	20		
Mode		W/R	EMG	ETS		
Frequency work	Hz	35		35		
Pulse duration	μS	250		250		
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	5	5	5		
Stim time	secs			5		
Rest time	secs	10	10	15		

Programs Urology

Progr.: 07		Mixed incontinence				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min		10			
Number of cycles		30				
Mode		EMG	W/R			
Frequency work	Hz		10			
Pulse duration	μ S		250			
Work time	secs	5	10			
Rest time	secs	10	5			
Progr.: 08		Urge incontinence				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		Cont				
Frequency work	Hz	10				
Pulse duration	μ S	250				
Progr.: 09		Stress- and Faecal-incontinence				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	30				
Pulse duration	μ S	200				
Ramp up time	secs	0,8				
Ramp down time	secs	0,8				
Work time	secs	5				
Rest time	secs	8				

Programs Urology

Progr.: 10	Mixed incontinence					
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	3	10	5	4	3
Mode		W/R	W/R	W/R	W/R	W/R
Frequency work	Hz	3	10	20	30	40
Pulse duration	μ S	250	250	250	200	200
Ramp up time	secs	0,8	0,8	0,8	0,7	0,7
Ramp down time	secs	0,8	0,8	0,8	0,7	0,7
Work time	secs	4	4	4	4	4
Rest time	secs	4	4	4	6	6
Progr.: 11		Pain 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	30				
Mode		Mod				
Frequency work	Hz	70/110				
Pulse duration	μ S	200/100				
Modulation time	secs	6				
Progr.: 12		Pain 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	30				
Mode		Han				
100 Hz / 200 μ S changing continuously with 2 Hz / 300 μ S in 3:3-second rhythm						
Progr.: PC1	Custom STIM					
Progr.: PC2	Custom EMG					
Progr.: PC3	Custom ETS					
Progr.: PC4	Custom STIM-EMG-ETS					

Programs Neurology

<i>Progr.: 01</i>		<i>EMG</i>				
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Mode</i>		<i>EMG</i>				
<i>Number of cycles</i>		30				
<i>Work time</i>	<i>secs</i>	8				
<i>Rest time</i>	<i>secs</i>	12				
<hr/>						
<i>Progr.: 02</i>		<i>ETS</i>				
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Mode</i>		<i>ETS</i>				
<i>Number of cycles</i>		25				
<i>Frequency work</i>	<i>Hz</i>	45				
<i>Pulse duration</i>	μ S	250				
<i>Ramp up time</i>	<i>secs</i>	1,5				
<i>Ramp down time</i>	<i>secs</i>	1,5				
<i>Work time</i>	<i>secs</i>	8				
<i>Stim time</i>	<i>secs</i>	4				
<i>Rest time</i>	<i>secs</i>	16				
<hr/>						
<i>Progr.: 03</i>		<i>Force 1</i>				
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Phase time</i>	<i>min</i>	5		5		
<i>Number of cycles</i>			20			
<i>Mode</i>		<i>W/R</i>	<i>EMG</i>	<i>Mod</i>		
<i>Frequency work</i>	<i>Hz</i>	45		8-12		
<i>Pulse duration</i>	μ S	250		300		
<i>Modulation time</i>	<i>secs</i>			10		
<i>Ramp up time</i>	<i>secs</i>	1				
<i>Ramp down time</i>	<i>secs</i>	1				
<i>Work time</i>	<i>secs</i>	6	6			
<i>Rest time</i>	<i>secs</i>	15	15			

Programs Neurology

<i>Progr.: 04</i>		<i>Force 2</i>				
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Phase time</i>	<i>min</i>	5		5		
<i>Number of cycles</i>			20			
<i>Mode</i>		<i>W/R</i>	<i>ETS</i>	<i>Mod</i>		
<i>Frequency work</i>	<i>Hz</i>	45	45	8-12		
<i>Pulse duration</i>	μ S	250	250	250		
<i>Modulation time</i>	<i>secs</i>			10		
<i>Ramp up time</i>	<i>secs</i>	1	1			
<i>Ramp down time</i>	<i>secs</i>	1	1			
<i>Work time</i>	<i>secs</i>	8	8			
<i>Stim time</i>	<i>secs</i>		4			
<i>Rest time</i>	<i>secs</i>	16	16			
<hr/>						
<i>Progr.: 05</i>		<i>Force 3</i>				
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Phase time</i>	<i>min</i>	5		5		5
<i>Number of cycles</i>			15		15	
<i>Mode</i>		<i>W/R</i>	<i>EMG</i>	<i>W/R</i>	<i>EMG</i>	<i>Mod</i>
<i>Frequency work</i>	<i>Hz</i>	45		45		8-12
<i>Pulse duration</i>	μ S	250		250		250
<i>Modulation time</i>	<i>secs</i>					10
<i>Ramp up time</i>	<i>secs</i>	1		1		
<i>Ramp down time</i>	<i>secs</i>	1		1		
<i>Work time</i>	<i>secs</i>	6	6	6	6	
<i>Rest time</i>	<i>secs</i>	15	15	15	15	
<hr/>						
<i>Progr.: 06</i>		<i>Force 4</i>				
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Phase time</i>	<i>min</i>	5			5	
<i>Number of cycles</i>			15	10		
<i>Mode</i>		<i>W/R</i>	<i>EMG</i>	<i>ETS</i>	<i>Mod</i>	
<i>Frequency work</i>	<i>Hz</i>	45		45	8-12	
<i>Pulse duration</i>	μ S	250		250	250	
<i>Modulation time</i>	<i>secs</i>				10	
<i>Ramp up time</i>	<i>secs</i>	1				
<i>Ramp down time</i>	<i>secs</i>	1				
<i>Work time</i>	<i>secs</i>	6	6	8		
<i>Stim time</i>	<i>secs</i>			4		
<i>Rest time</i>	<i>secs</i>	15	15	16		

Programs Neurology

Progr.: 07		Relaxation				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5		5		
Number of cycles			20			
Mode		Mod	EMG	Mod		
Frequency work	Hz	8-12		8-12		
Pulse duration	μS	250		250		
Modulation time	secs	10		10		
Work time	secs		20			
Rest time	secs		10			
Progr.: 08		Rehabilitation 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	50				
Pulse duration	μS	300				
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	10				
Rest time	secs	15				
Progr.: 09		Rehabilitation 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5	15	10		
Mode		Cont	W/R	Mod		
Frequency work	Hz	5	60	8-12		
Pulse duration	μS	300	350	250		
Modulation time	secs			6		
Ramp up time	secs		2			
Ramp down time	secs		1,5			
Work time	secs		7			
Rest time	secs		14			

Programs Neurology

Progr.: 10	<i>Partial peripheral paralysis</i>					
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Phase time</i>	<i>min</i>	10				
<i>Mode</i>		<i>W/R</i>				
<i>Frequency work</i>	<i>Hz</i>	60				
<i>Pulse duration</i>	μ S	450				
<i>Ramp up time</i>	<i>secs</i>	0,6				
<i>Ramp down time</i>	<i>secs</i>	1				
<i>Work time</i>	<i>secs</i>	4				
<i>Rest time</i>	<i>secs</i>	20				
Progr.: 11	<i>Pain 1</i>					
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Phase time</i>	<i>min</i>	30				
<i>Mode</i>		<i>Mod</i>				
<i>Frequency work</i>	<i>Hz</i>	70/110				
<i>Pulse duration</i>	μ S	200/100				
<i>Modulation time</i>	<i>secs</i>	6				
Progr.: 12	<i>Pain 2</i>					
		<i>Phase 1</i>	<i>Phase 2</i>	<i>Phase 3</i>	<i>Phase 4</i>	<i>Phase 5</i>
<i>Phase time</i>	<i>min</i>	30				
<i>Mode</i>		<i>Han</i>				
<i>100 Hz / 200 μS changing continuously with 2 Hz / 300 μS in 3:3-second rhythm</i>						
Progr.: PC1	<i>Custom STIM</i>					
Progr.: PC2	<i>Custom EMG</i>					
Progr.: PC3	<i>Custom ETS</i>					
Progr.: PC4	<i>Custom STIM-EMG-ETS</i>					

Programs Urology + Neurology

Progr.: 01	EMG - Urology					
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Mode		EMG				
Number of cycles		40				
Work time	secs	8				
Rest time	secs	8				
Progr.: 02	EMG - Neurology					
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Mode		EMG				
Number of cycles		30				
Work time	secs	8				
Rest time	secs	12				
Progr.: 03	ETS - Urology					
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Mode		ETS				
Number of cycles		25				
Frequency work	Hz	35				
Pulse duration	μ S	300				
Ramp up time	secs	1,5				
Ramp down time	secs	1,5				
Work time	secs	8				
Stim time	secs	5				
Rest time	secs	12				
Progr.: 04	ETS - Neurology					
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Mode		ETS				
Number of cycles		25				
Frequency work	Hz	45				
Pulse duration	μ S	250				
Ramp up time	secs	1,5				
Ramp down time	secs	1,5				
Work time	secs	8				
Stim time	secs	4				
Rest time	secs	16				

Programs Urology + Neurology

Progr.: 05		Stress 1 - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5				
Number of cycles			25			
Mode		W/R	EMG			
Frequency work	Hz	35				
Pulse duration	μS	300				
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	5	5			
Rest time	secs	10	10			
Progr.: 06		Stress 2 - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5				
Number of cycles			30			
Mode		W/R	ETS			
Frequency work	Hz	35	35			
Pulse duration	μS	300	300			
Ramp up time	secs	1	1			
Ramp down time	secs	1	1			
Work time	secs	5	5			
Stim time	secs		5			
Rest time	secs	15	15			
Progr.: 07		Stress 3 - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5		5		
Number of cycles			20		20	
Mode		W/R	EMG	W/R	EMG	
Frequency work	Hz	35		35		
Pulse duration	μS	250		250		
Ramp up time	secs	1		1		
Ramp down time	secs	1		1		
Work time	secs	5	5	5	5	
Rest time	secs	10	10	10	10	

Programs Urology + Neurology

Progr.: 08		Stress 4 - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5				
Number of cycles			20	20		
Mode		W/R	EMG	ETS		
Frequency work	Hz	35		35		
Pulse duration	µS	250		250		
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	5	5	5		
Stim time	secs			5		
Rest time	secs	10	10	15		
Progr.: 09		Mixed incontinence - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min		10			
Number of cycles		30				
Mode		EMG	W/R			
Frequency work	Hz		10			
Pulse duration	µS		250			
Work time	secs	5	10			
Rest time	secs	10	5			
Progr.: 10		Force 1 - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5		5		
Number of cycles			20			
Mode		W/R	EMG	Mod		
Frequency work	Hz	45		8-12		
Pulse duration	µS	250		300		
Modulation time	secs			10		
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	6	6			
Rest time	secs	15	15			

Programs Urology + Neurology

Progr.: 11		Force 2 - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5		5		
Number of cycles			20			
Mode		W/R	ETS	Mod		
Frequency work	Hz	45	45	8-12		
Pulse duration	μ S	250	250	250		
Modulation time	secs			10		
Ramp up time	secs	1	1			
Ramp down time	secs	1	1			
Work time	secs	8	8			
Stim time	secs		4			
Rest time	secs	16	16			
Progr.: 12		Force 3 - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5		5		5
Number of cycles			15		15	
Mode		W/R	EMG	W/R	EMG	Mod
Frequency work	Hz	45		45		8-12
Pulse duration	μ S	250		250		250
Modulation time	secs					10
Ramp up time	secs	1		1		
Ramp down time	secs	1		1		
Work time	secs	6	6	6	6	
Rest time	secs	15	15	15	15	
Progr.: 13		Force 4 - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5			5	
Number of cycles			15	10		
Mode		W/R	EMG	ETS	Mod	
Frequency work	Hz	45		45	8-12	
Pulse duration	μ S	250		250	250	
Modulation time	secs				10	
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	6	6	8		
Stim time	secs			4		
Rest time	secs	15	15	16		

Programs Urology + Neurology

Progr.: 14		Relaxation - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5		5		
Number of cycles			20			
Mode		Mod	EMG	Mod		
Frequency work	Hz	8-12		8-12		
Pulse duration	μ S	250		250		
Modulation time	secs	10		10		
Work time	secs		20			
Rest time	secs		10			
Progr.: 15		Urge incontinence - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		Cont				
Frequency work	Hz	10				
Pulse duration	μ S	250				
Progr.: 16		Stress- and Faecal-incontinence - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	30				
Pulse duration	μ S	200				
Ramp up time	secs	0,8				
Ramp down time	secs	0,8				
Work time	secs	5				
Rest time	secs	8				
Progr.: 17		Mixed incontinence - Urology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	3	10	5	4	3
Mode		W/R	W/R	W/R	W/R	W/R
Frequency work	Hz	3	10	20	30	40
Pulse duration	μ S	250	250	250	200	200
Ramp up time	secs	0,8	0,8	0,8	0,7	0,7
Ramp down time	secs	0,8	0,8	0,8	0,7	0,7
Work time	secs	4	4	4	4	4
Rest time	secs	4	4	4	6	6

Programs Urology + Neurology

Progr.: 18		Rehabilitation 1 - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	50				
Pulse duration	μ S	300				
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	10				
Rest time	secs	15				
Progr.: 19		Rehabilitation 2 - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5	15	10		
Mode		Cont	W/R	MOD		
Frequency work	Hz	5	60	8-12		
Pulse duration	μ S	300	350	250		
Modulation time	secs			6		
Ramp up time	secs		2			
Ramp down time	secs		1,5			
Work time	secs		7			
Rest time	secs		14			
Progr.: 20		Partial peripheral paralysis - Neurology				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	10				
Mode		W/R				
Frequency work	Hz	60				
Pulse duration	μ S	450				
Ramp up time	secs	0,6				
Ramp down time	secs	1				
Work time	secs	4				
Rest time	secs	20				

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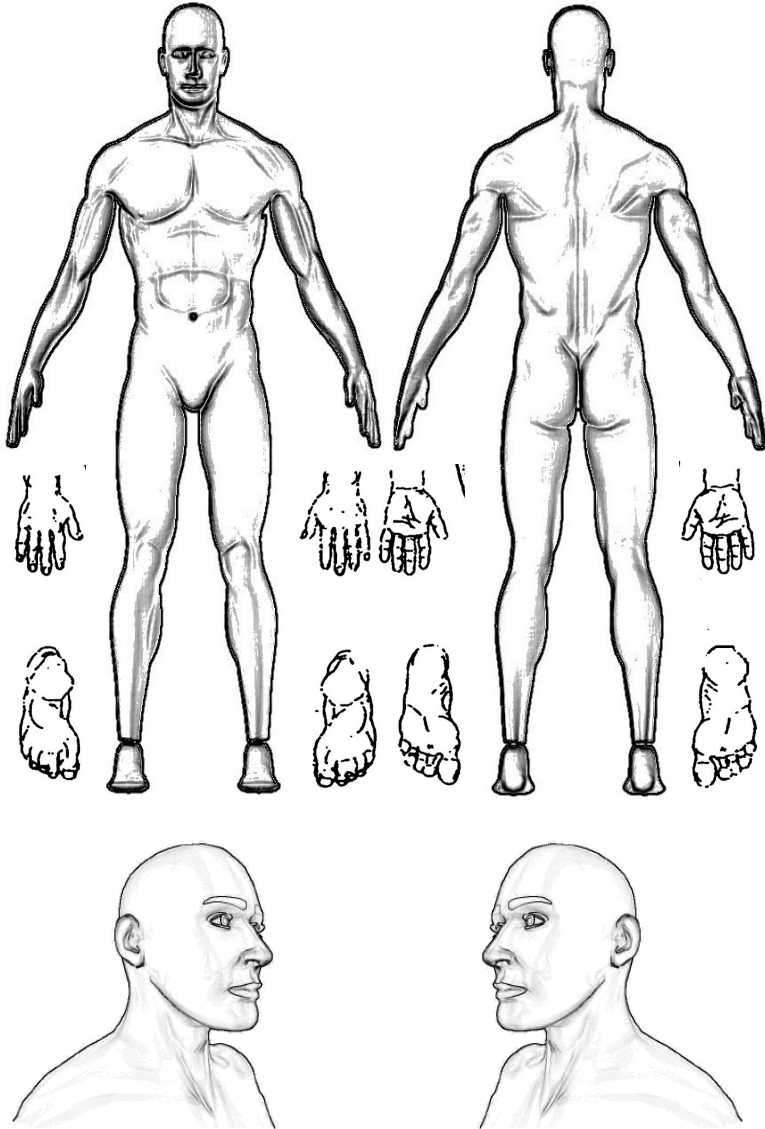
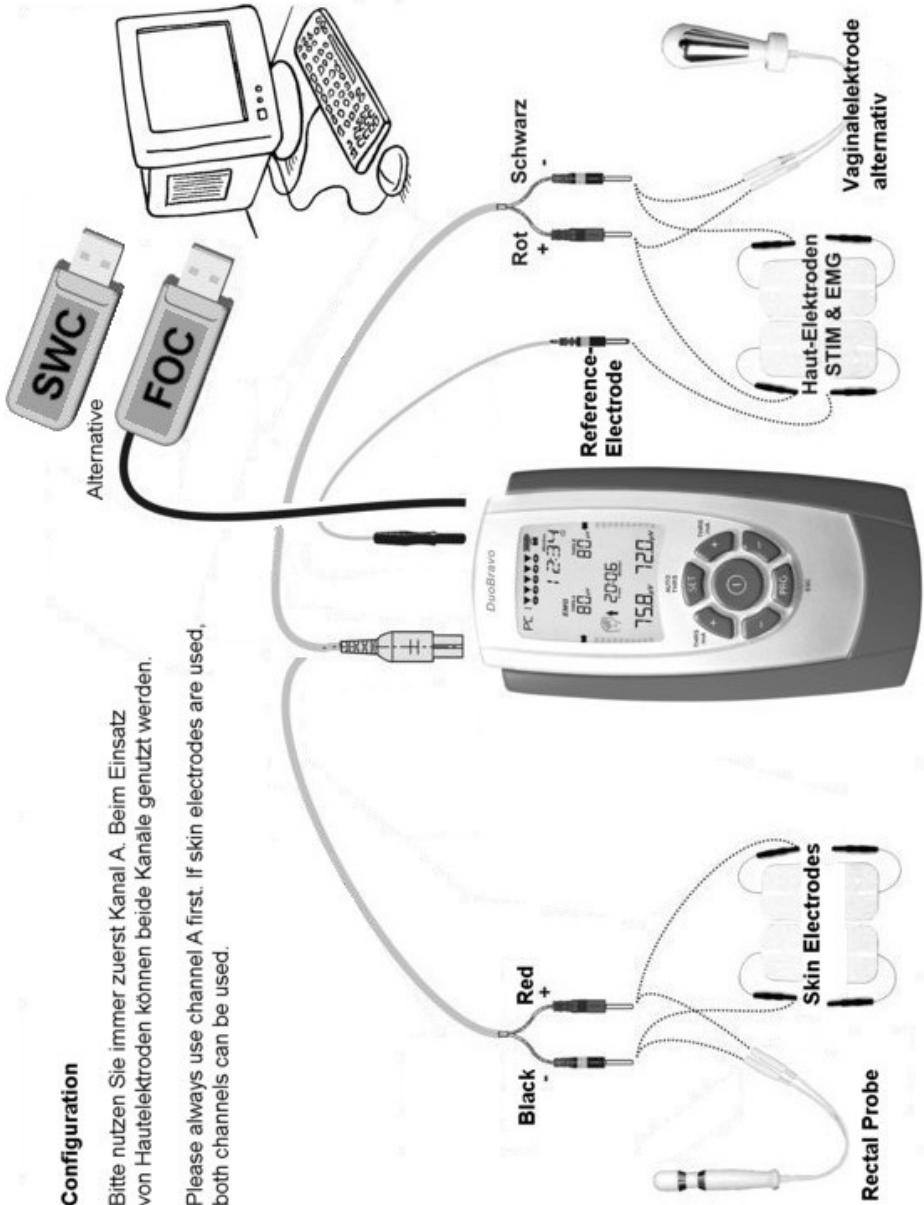
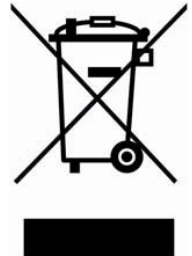
In accordance with European regulations [1], used electrical and electronic equipment may not be discarded in household or unsorted municipal waste. Use separate collection facilities. Contact your local government or council for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the ground water and get into the food chain damaging the health and well being of people and animals.

[1] European Community directive 2012/19/EU on waste electrical and electronic equipment.

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The batteries should be replaced when they are drained or stop functioning. The batteries supplied with the **DuoBravo** are recyclable. Remove the batteries from the unit and follow your local recycling guidelines.

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A list of literature regarding the topics of this manual can be found on our website „www.mtrplus.com“.

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Reserved for changes.

Printing or copy of this manual - also in an abridged version - is forbidden and can only be allowed with a written approval of “MTR+ Vertriebs GmbH”.

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Electromagnetic Compatibility

If the device is used according to the instructions given in this manual, it corresponds to the safety requirements for medical electrical systems IEC 60601-1. **DuoBravo** was checked and is in accordance with the limits set for medical products in the standard IEC 60601-1-2:2014. These limits serve to the safety of the user regarding electromagnetic disturbances. The **DuoBravo** is a Class B device and is suitable for home healthcare environments as well as typical clinical environments.

Guidance and manufacturer's declaration - electromagnetic emissions		
The DuoBravo unit is intended for use in the electromagnetic environment specified below. The customer or the user of the DuoBravo should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The DuoBravo uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The DuoBravo is suitable for use in all establishments, including domestic and medical establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not Applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not Applicable	

Please adhere to these warnings when using the device:

Don't use **DuoBravo** near active HF surgical equipment or RF shielded rooms used for magnetic resonance imaging, where the intensity of EM disturbances is high.

Use of this equipment adjacent to, or placement on top of other equipment should be avoided in case output or EMG reading of the **DuoBravo** is effected. If this is necessary, Both the **DuoBravo** and the other equipment should be monitored to verify that they are both operating normally.

Use of accessories and lead wires other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used not closer than 4 meters to any part of the **DuoBravo** in EMG or ETS mode or 30 cm in Stimulation mode, including lead wires specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Electromagnetic Compatibility

Guidance and manufacturer's declaration - electromagnetic Immunity			
The DuoBravo is intended for use in the electromagnetic environment specified below. The customer or the user of the DuoBravo should assure that it is used in such an environment			
Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %
Electrical fast transient/burst IEC 61000-4-4	Power supply lines: ±2 kV input/output lines: ±1 kV	Not Applicable	DuoBravo is powered by batteries
Surge IEC 61000-4-5	line(s) to line(s): ±1 kV. line(s) to earth: ±2 kV. 100 kHz repetition frequency	Not Applicable	DuoBravo is powered by batteries
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° phases 0% U_T for 0.5 cycle And at single phase 0° 0% U_T for 1 cycle 70% U_T for 25/30 cycles 0% U_T for 250 cycles 0% U_T for 300 cycles	Not Applicable	DuoBravo is powered by batteries
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC61000-4-6	150KHz to 80MHz: 3Vrms 6Vrms (in ISM and amateur radio bands)	Not Applicable	DuoBravo is powered by batteries
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1: U_T is the a.c. mains voltage prior to application of the test level. NOTE 2: At 80 MHz the higher frequency range applies. NOTE 3: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

Electromagnetic Compatibility

Guidance and manufacturer's declaration - electromagnetic Immunity


The **DuoBravo** is intended for use in the electromagnetic environment specified below. The customer or the user of the **DuoBravo** should assure that it is used in such an environment

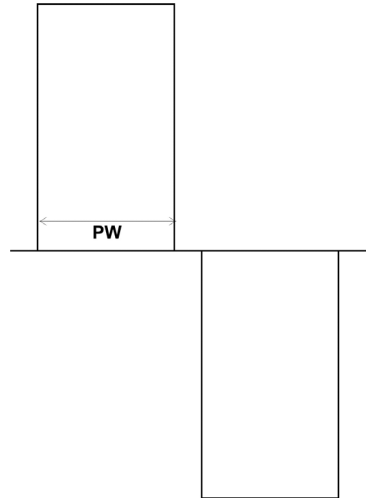	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380 – 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27
	450	380 – 390	GMRS 460, FRS 460	FM c) ± 5 kHz deviation 1 kHz sine	2	0,3	28
	710	704 – 787	LTE Band 13, 17	Pulse modulation 217 Hz	0,2	0,3	9
	745						
	780						
	810	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0,3	28
	870						
	930						
	1720	1 700 – 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0,3	28
	1845						
	1970						
	2450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0,3	28
	5240	5 100 – 5 800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0,2	0,3	9
	5240						
	5785						

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